

ITEM# B3200



DELUXE ALUMINUM SHOWER BENCH

300 lb Weight Capacity



Rhythm Healthcare, LLC
3200 Tyrone Blvd N
St. Petersburg, FL 33710

contactus@rhythmhc.com
Website: www.rhythmhc.com
(877) 843-6464

V1: 5/4/23

GENERAL EXPLANATION OF SHOWER BENCH



USER GUIDE

⚠ Prior to use, please read all instructions.
We recommend you keep this user guide for future reference.

SET UP

Bench Assembly

- 1 Slide the narrow end of each leg into one of the leg openings on the base. Push and turn until chrome button is completely through the hole on the leg opening. This should make a "click" sound.
- 2 Adjust height using numbered holes and chrome push button. Turn stool over and test for stability. If unstable, check to make sure all legs are at the same height.



Adjusting Leg Height

- 1 Adjust the leg height by pushing the button and sliding the tube up or down. The height will adjust one hole at a time.
- 2 Ensure that all legs are adjusted to the same height.



CLEANING



Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

> Use a non-abrasive detergent or household cleaner with warm water.

> Rinse with warm water and wipe dry.

> DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your product or warranty please call your local dealer or contact us directly.