

CUP HOLDER - 9ACO2





- 1. Place part (A) around frame at the desired position.
- 2. Take out part (B) rubber insert if necessary for cup holder to fit to tubing width.
- 3. Place screw through holes, thread and fasten using knob part (C). Move cup to your preferred position then tighten the knob.
- 4. DO NOT over tighten.

Rhythm Healthcare, LLC Clearwater, FL 33762

contactus@rhythmhc.com www.rhythmhc.com One Year Limited warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For Warranty Service, it is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare will determine if the product will be repaired or replaced