

LIGHT HOLDER - 9ACO4



Figure 1



Figure 2



Figure 3

TOOL FREE
ASSEMBLY

SET UP

Requires 2 CR 2032 batteries (not included).

1. Using a flat head screwdriver or your finger, using some force, remove the cover on the left side of the rollator (facing front as the user). Figure 1.

2. To insert two CR 2032 batteries (not included), unscrew head of light. Paying attention to how it goes back together, remove black ring, glass and light bulb. Pull the battery holder using the metal clip. Insert batteries in the black battery holder. Reassemble.

3. Place the light in the receptacle. Figure 2.

4. Adjust the light to face desired direction.

5. To turn the light on, press the green button on the back of the light. Figure 3.

Rhythm Healthcare, LLC
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com

One Year Limited warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For Warranty Service, it is recommended but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.