



ITEM# B3250HDWH

DELUXE BARIATRIC BATH BENCH WITH CROSS-FRAME BRACE

WEIGHT CAPACITY: 500 LBS



Rhythm Healthcare
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com
(877) 843-6464

V2: 6/22/23

PARTS DIAGRAM

⚠ Read entire manual prior to assembly and use and save this manual for future reference.



SAFETY PRECAUTIONS

- › Assistance is required for users with limited physical capabilities.
- › Ensure all Spring Pins and U-Clips protrude fully through setting holes before use.
- › Check that shower chair is stable and positioned properly.
- › Check leg tips for rips, cracks or other wear and replace if necessary.

MAINTENANCE

⚠ Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Clean with mild soap and rinse with warm water and wipe dry after each use.
- › Inspect weekly for wear and if any component appears worn or is not working properly, contact your dealer for repair or appropriate replacement parts.

ASSEMBLY INSTRUCTIONS


⚠ If components are damaged or missing, contact your dealer immediately. Do not substitute parts.

Required tools: Phillips head screw driver and a wrench

Assembling the Cross Brace

Take the upper and lower Cross Braces Assemblies and place the flat middle areas together. Insert the Wing Bolt through the Upper Cross Brace into the Lower Cross Brace and screw on the Bolt Cap but do not tighten.

Insert each Leg into the lower tubes of the Cross Brace making sure the Leg Tips are angled outward. Set each Leg at same height using the U-Clips. Make sure the pins of the U-Clips fully protrude through the setting holes.

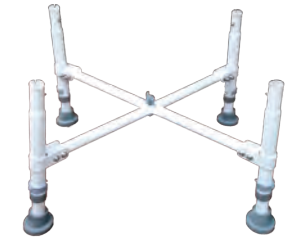

Adding the Seat Assembly

While pushing in the Spring Pins on the upper tubes of the Cross Brace, insert them into the tubes of the Seat Assembly until the Spring Pins click into place. Then tighten the Wing Bolt on the Cross Brace.

Adding the Back Rest

While pushing in the Spring Pins on the long ends of the Back Rest Attachment Tubes, insert them into the receiving brackets on the bottom side of the Seat Assembly until the Spring Pins click into place. Then push in the Spring Pins on the Back Rest Assembly tubes and insert them into the Back Rest Attachment Tubes until the Spring Pins click into place.

Adjusting Leg Height

Remove the U-Clip, slide the leg up or down to desired height and replace the U-Clip making sure the pin protrudes fully through the setting hole. Repeat on all legs making sure to set all to same height.

LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended, but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare prior authorization is to be required. In the event of a defect covered by this Warranty, Rhythm Healthcare will determine if the product will be repaired or replaced. Rhythm Healthcare products are identified by a serial or a lot number label at the time of the manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare product or this warranty, please email us at contactus@rhythmhc.com