



## Half Length Bed Rails

Some Additional Products:




Full Electric Bed



Transfer Bench



Shower Sprayer

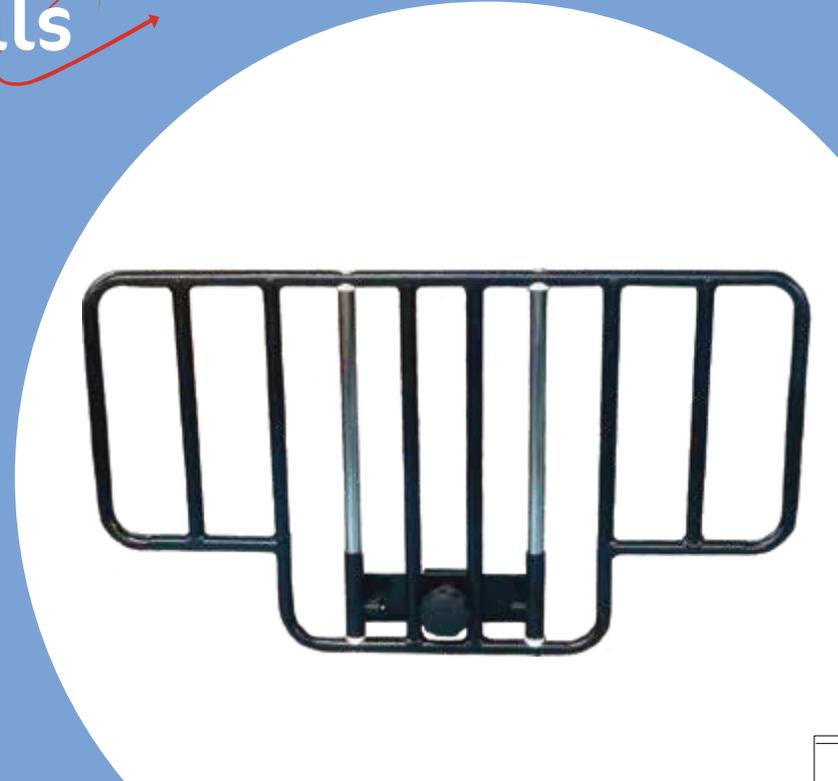

LIFESTYLE MOBILITY AIDS  
Email : [contactus@lifestylemobilityaids.com](mailto:contactus@lifestylemobilityaids.com)  
Website : [www.lifestylemobilityaids.com](http://www.lifestylemobilityaids.com)

H6500H

# USER MANUAL - H6500H HALF LENGTH BED RAILS

## GENERAL EXPLANATION



Spring Loaded Knob to raise and lower Bed Rail

Screw Knob to attach Bed Rail to bed frame

## SPECIFICATIONS

RAIL WIDTH	32"	WEIGHT	9 lbs each
RAIL HEIGHT	18" (16" above deck)	SOLD IN PAIRS	

## ASSEMBLY INSTRUCTIONS

### Installing Half Length Bed Rail

NOTE: Can be installed at head or foot end of bed.

- 1 Lower bed to flat position and remove mattress.
- 2 Loosely assemble upper Clamp (Figure 1) placing spacer between the Screw Knob and front side of the Bed Rail.
- 3 Hook Fixed Lower Clamp under outside edge of bed frame. (Figure 2)
- 4 Hook Upper Clamp on top of bed frame and tighten Screw Knob until secure. (Figure 3)



FIGURE 1  
Upper Clamp & Screw Knob

NOTE: Wiggle the bed rail slightly while tightening so that the upper and lower clamps are properly seated.



FIGURE 2  
Fixed Lower Clamp & Moveable Upper Clamp



FIGURE 3  
Installed Bed Rail


2

## OPERATING INSTRUCTIONS

- ⚠ Visit <http://www.fda.gov> to view the "FDA Entrapment Zones and Guidelines" to learn about the risks of entrapment.

### Adjusting Half Length Bed Rail

- 1 To raise or lower the Bed Rail, pull and hold the Spring loaded Knob while sliding Bed Rail up or down.
- 2 Check Bed Rail weekly to ensure that Screw Knob is secure.

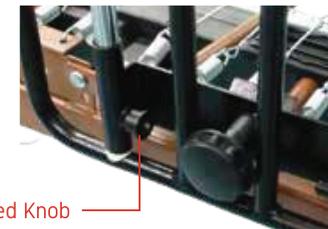

Spring Loaded Knob

## SAFETY PRECAUTIONS

- ⚠ Read entire manual prior to assembly and operation and save for future reference.
- › DO NOT install or use this equipment without first reading and understanding these instructions. If you do not understand these instructions, contact your dealer or a health care professional before attempting to install and use this equipment – Improper installation and/or use of this product can result in entrapment, serious injury or death.
- › Lifestyle Mobility Aids is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.
- › The Bed Rails are not to be used as bedside assist handles when getting in or out of bed and can be damaged if excessive pressure is applied.
- › The Bed Rails are to prevent the user from inadvertently rolling out of bed, NOT for restraint purposes. If restraint is needed, consult a physician or healthcare professional.
- › DO NOT use the Bed Rails as push handles when moving the bed.
- › Visit <http://www.fda.gov> to view the "FDA Entrapment Zones and Guidelines" to learn about the risks of entrapment.

## MAINTENANCE

- › Clean weekly with soft cloth using mild soap or detergent
- › DO NOT use organic solvents, such as acetone, or abrasive cleansers.
- › Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

## WARRANTY

Limited Lifetime warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

It is recommended but not required that the product be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However, all Lifestyle Mobility Aids Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Lifestyle Mobility Aids, prior authorization will be required. In the event of a defect covered by this warranty, Lifestyle Mobility Aids will determine if the product will be repaired or replaced.

This Lifestyle Mobility Aids product is identified by a serial or lot number at the time of manufacture. Removing or defacing the serial or lot number label may void this warranty. If you have questions about your Lifestyle Mobility Aids product or this warranty, please contact us at [contactus@lifestylemobilityaids.com](mailto:contactus@lifestylemobilityaids.com).

3