

## LM5BA SETUP

# This setup checklist should be used for patients receiving a LM5BA Device



## Placing the Concentrator in the Home

Choose a location in the home where there is airflow around the device (avoid closets, behind Curtains, etc.)

Show the patient how to plug the device into the wall, no extension cords should be used with the device.

## Getting started with the Stationary Oxygen Concentrator

Show the patient the on/off switch.

Show the patient how to operate the Flow Meter and set the flow to the patient prescription.

Demonstrate how to attach the Cannula to the Concentrator.

If prescribed, demonstrate the operation and attachment of the humidifer bottle. Advise on distilled water and cleaning instructions per manual.

#### Filters and Maintenance

Show location of Sponge Filter, show patient how to remove and replace Filter from LM5BA.

Provide patient with spare Sponge Filter and explain cleaning instructions for the Sponge Filter.

#### **Alarms**

Review device alarms with patient.

Demonstrate how to silence/turn-off alarm.

## Back-up Oxygen Plan

Show patient back-up oxygen plan in case of concentrator or power failure.

## **Patient Safety Information**

Provide patient with no smoking sign for home.

Educate patient on smoke infiltration and device warranty limitations.

Provide patient guidance on not using oxygen while cooking or near open flames.

#### **Patient Leave Behinds**

Provide patient with user manual.

Provide patient with emergency contact information and when to call 911.