

Portable Oxygen Concentrator P2-E6

GETTING STARTED GUIDE

Now that you have unpacked each box, you have everything you need to start using your P2-E6. The set-up of your P2-E6 is quick and simple.

Before using the P2-E6, please read and understand the entire user manual.

If you have any questions, please contact your provider before use.



Simple Steps to Independence

1



Find a well ventilated location to place the P2-E6. Be sure to keep the Intake and exhaust with clear access.

3



Plug the power supply AC cord into the nearest electrical outlet.

5



Press the ON/OFF button, device will beep and the indicator light will flash. A two minute warm-up time will initiate.

2



Slide the flat side of the battery until the latch returns to the upper position and you hear it click into place.

4




Connect the AC power to the P2-E6. The green LED on the power adapter will be on and the concentrator will beep.

6




Connect the nasal cannula to the nozzle fitting at the top of the P2-E6. Ensure that it is routed to prevent it from being pinched or kinked to avoid disruption of oxygen flow.

7

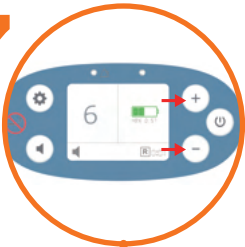

Change the P2-E6 to the flow setting prescribed by your physician. Press the + or - buttons to adjust to the desired setting.

8

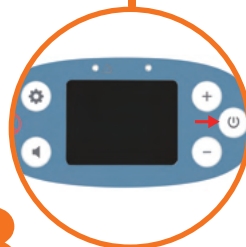

TO SHUT DOWN:
Press and hold the ON/OFF button for 3 seconds.

Additional information
When adapter is connected/disconnected or battery is installed/uninstalled, device will beep and display will show status information for battery charge.

Intake Filter - Item # P2FC-1

In normal use conditions, the air filter must be replaced after approximately 6 months of daily use. When conditions are subject to higher levels of dust or dirt, we recommend periodically checking the air filter and if the filter shows a gray or brown color, replace it.

The intake filter is designed to ensure clean air enters the compressor. To replace the intake filter:

1. Unscrew the bottom of the pre-filter with a Philips screwdriver.
2. Lift up the pre-filter from the bottom end, then remove it.
3. Remove the intake filter from the intake chamber.
4. Insert a new intake filter into the chamber;
5. Reinstall the pre-filter.

Replacement pre-filters and intake filters can be purchased from your equipment provider.

Battery Care and Maintenance

The P2-E6 Lithium Ion Battery requires special care to ensure proper performance and long life. Use only P2-E6 batteries # P2BY001-2 with your concentrator.



CAUTION

Keep liquids away from batteries.

If batteries become wet, stop use immediately and dispose of battery properly.

Battery Replacement

2. Press down the latch, and slide the battery out.



2. Insert the new P2-E6 battery by sliding battery into place until the latch returns to the upper position.





Setting	Battery Life
1	5.4h
2	4.8h
3	3.3h
4	2.5h
5	2.1h
6	1.7h

Store battery in a cool, dry place. Store with a charge of 40-50%.

Batteries should not be left dormant for more than 90 days at a time.





Rhythm Healthcare, LLC
contactus@rhythmhc.com
www.rhythmhc.com