

Portable Oxygen Concentrator P2

GETTING STARTED GUIDE

Now that you have unpacked each box, you have everything you need to start using your P2. The set-up of your P2 is quick and simple.

Before using the P2, please read and understand the entire user manual.

If you have any questions, please contact your provider before use.



Simple Steps to Independence

1



Find a well ventilated location to place the P2. Make sure intake and exhaust fan is clear and nothing is blocking it.

3



Plug the AC cord into the nearest electrical outlet.

5



Press the ON/OFF button for one second. It will beep and the indicator light will flash. Wait two minutes for the machine to warm up.

2



Slide flat side of the battery into place until the latch returns to the upper position and you hear it click into place.

4



Connect the AC connector to the P2. The green LED on the power adapter will be on and the concentrator will beep.

6



Join the nasal cannula to the nozzle fitting. Ensure that the cannula is routed to prevent it from being pinched or kinked to avoid a disruption of oxygen flow.

7



Change the P2 Concentrator setting to the flow rate prescribed by your physician or clinician.

Press the + or – setting buttons for the desired setting.

8



TO SHUT DOWN:
Press and hold the ON/OFF button for 3 seconds.

ADDITIONAL INFORMATION:
When adapter is connected/disconnected or battery is installed/uninstalled, unit will beep and the display will show power supply information.

Intake Filter - Item # P2FC-1

In normal use conditions, the air filter must be replaced after approximately 6 months of daily use. When conditions are subject to higher levels of dust or dirt, we recommend periodically checking the air filter and if the filter shows a grey or brown color, replace it.

The intake filter is designed to ensure clean air enters the compressor. To replace the intake filter:

1. Unscrew the bottom of the pre-filter with a Philips screwdriver.
2. Lift up the pre-filter from the bottom end, then remove it.
3. Remove the intake filter from the intake chamber.
4. Insert a new intake filter into the chamber;
5. Reinstall the pre filter.

Replacement pre-filters and intake filters can be purchased from your equipment provider.

Battery Care and Maintenance

The P2 Lithium Ion Battery requires special care to ensure proper performance and long life. Use only P2 batteries #P2BY001-1 (compatible with batteries with Lifestyle logo) and P2BY001-2 (compatible with batteries with Rhythm logo) with your concentrator.



CAUTION

Keep liquids away from batteries.

If batteries become wet, stop use immediately and dispose of battery properly.

Battery Replacement

2. Press down the latch, and slide the battery out.



2. Insert the new P2 battery by sliding battery into place until the latch returns to the upper position.





Battery Duration Chart

Setting	Duration
1	5h 00min
2	4h 00min
3	3h 00min
4	2h 14min
5	1h 50min

Store battery in a cool, dry place. Store with a charge of 40-50%.

Batteries should not be left dormant for more than 90 days at a time.





Rhythm Healthcare, LLC
contactus@rhythmhc.com
www.rhythmhc.com