

## LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame. One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and/or unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your provider for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare, LLC provider through whom it was originally purchased. However, all Rhythm Healthcare, LLC providers are qualified to assist you in obtaining warranty service. Prior authorization is required for products returned to Rhythm Healthcare, LLC.

All Rhythm Healthcare, LLC products are identified by a serial number at the time of manufacture. Removing or defacing the serial number may void this warranty. In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.

For questions, please email us at [Contactus@rhythmhc.com](mailto:Contactus@rhythmhc.com).

Visit [www.rhythmhc.com](http://www.rhythmhc.com) for more information about our products and accessories.



ITEM#: 729

**RHYTHM**  
HEALTHCARE

## RALLY LITE 3 WHEEL ROLLATOR

250 lb  
Weight Capacity



## USER INSTRUCTIONS

Please read instructions before assembly and use of this product. This rollator must be assembled and fitted for the user by a qualified healthcare professional or provider.

Unfold the rollator by pulling side frames apart. Then push center folding mechanism toward front of rollator to lock in place. Extend rear wheels by sliding rear frame tubing out until push pins lock into place. Insert screw knobs on each side and tighten securely.

Insert handles into receiving tubes at top of frame with brake cables to outside of frame. Adjust handles to desired height and insert screw knobs to lock into place. Tighten screw knobs securely and attach brake cables to clips on the frame.

Lock the brakes by pushing the brake handles down until they click into place and the rear wheels do not roll.

To fold for storage, pull back on the upper folding mechanism and push the handles together. To unfold for use, pull hand grips apart and push folding mechanism until it locks into place.

## CARE & MAINTENANCE

- Ensure that all hardware is secure at all times.
- Replace any broken, damaged and/or worn parts immediately.
- Clean with warm water and a mild cleanser. Dry with a clean, soft cloth.
- Brakes may loosen over time and should be adjusted by a qualified provider. Failure to do this may result in injury to the user.

## PRECAUTIONS

- Do NOT use before extending frame with rear wheels fully.
- Watch out for cords, throw rugs, toys on the floor and/or spilled liquids.
- Make sure all folding mechanisms are in the locked position when the walker is in use.
- Do NOT use on stairs or curbs.

**RHYTHM**  
HEALTHCARE

Rhythm Healthcare, LLC  
Clearwater, FL 33762

(877) 843-6464  
[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
[www.rhythmhc.com](http://www.rhythmhc.com)