

## LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. Prior authorization is required for walkers returned to Rhythm Healthcare.

All Rhythm Healthcare products are identified by a serial number at the time of manufacture. Removing or defacing the serial number may void this warranty.

If you have questions about your rollator or this warranty, please call (877) 843-6464.

Visit [www.rhythmhc.com](http://www.rhythmhc.com) for more information about our products and accessories.



## ROYAL SERIES ROLLATORS

300 lb  
Weight Capacity



## OPERATING INSTRUCTIONS

Our Royal Series Rollator is available in both aluminum and steel with a weight capacity of 300 lbs.

Unfold the rollator and insert the handles into the top of the frame with the brake cables on the outside of the frame. Then, clip the brake cable onto the back frame, close to the cross bar.

For proper adjustment of the handles, extend your arm toward the floor. The hand grip should be at wrist height. With your hand on the grip, your arm should be flexed 20° - 30°. Be sure to adjust both handles to the same height.

When using the rollator as a seat, the brakes must be in the locked position. Lock the brakes by pushing the brake handles down until they click into place and the rear wheels do not roll.

To fold for storage, lift the seat and pull up on the bar beneath the center of the tote bag.

To unfold, push down on the hand grips.

## MAINTENANCE

- Ensure that all hardware is secure at all times.
- Replace any broken, damaged and/or worn parts immediately.
- Clean wheels with warm water and a mild cleanser. Dry with a clean, soft cloth.
- Brakes may loosen over time and should be adjusted by a qualified dealer.

Failure to do this may result in injury to the user.

## PRECAUTIONS

- Do NOT use on stairs or curbs.
- Watch out for cords, throw rugs, toys on the floor and/or spilled liquids.
- Make sure all folding mechanisms are in the locked position when the walker is in use.
- Engage brakes when using seat.
- Do NOT use as a wheelchair.

Rhythm Healthcare  
Clearwater, FL 33762

[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)  
(877) 843-6464

V4: 7/27/23