

## LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and tires are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. Prior authorization is required for walkers returned to Rhythm Healthcare, LLC.

All Rhythm Healthcare, LLC products are identified by a serial number at the time of manufacture. Removing or defacing the serial number may void this warranty.

If you have questions about your rollator or this warranty, please call (877) 843-6464.

Visit [www.rhythmhc.com](http://www.rhythmhc.com) for more information about our products and accessories.

ITEM #: 807SN



## 4-WHEEL ROLLATOR

300 lb  
Weight Capacity



## ASSEMBLY INSTRUCTIONS

Please read and understand manual before operating the rollator.

1. Open frame and slide backrest into frame brackets. Ensure it clicks into place and push pins buttons are visible in hole.
2. Insert handle tube into frame at desired height. Ensure each side is at the same height.
3. Slide bolts through holes from the inside and screw on tightening knob until secure.

If brakes need to be tightened or adjusted, please bring them to a qualified dealer.

## OPERATING INSTRUCTIONS

1. Ensure rollator is fully opened, seat is down and the folding bar under seat is locked.
2. Test rollator for stability before leaning on with full weight.
3. Test loop locks before operating. If the wheels roll while brakes are locked, the rollator should be taken to dealer for adjustment. When locks are released, the wheels should roll freely.
4. Always lock loop locks before sitting. Only use seat when the rollator is on an even and stable surface.

### Using the loop brakes

1. To Slow or stop, with palm/hand wrapped around handle, squeeze the brake slowly, do not engage too quickly - as this can cause the rollator to stop abruptly.
2. To lock wheels, push the loop locks down, towards the ground, and you will hear them click into place.
3. To release wheel locks, push up both loop locks.

## MAINTENANCE

- Ensure that all hardware is secure at all times.
- Replace any broken, damaged and/or worn parts immediately.
- Clean wheels with warm water and a mild cleanser. Dry with a clean, soft cloth.
- Brakes may loosen over time and should be adjusted by a qualified dealer.

## PRECAUTIONS

- Do NOT use on stairs or curbs.
- Watch out for cords, throw rugs, toys on the floor and/or spilled liquids.
- Make sure all folding mechanisms are in the locked position when the walker is in use.
- Engage brakes when using seat.
- Do NOT use as a wheelchair.

Rhythm Healthcare  
Clearwater, FL 33762

[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)  
(877) 843-6464

V2: 9/13/23