



955CH Arpeggio
(Champagne)

956BK Arpeggio
(tall, Black)



ARPEGGIO ROLLATOR ARPEGGIO XT ROLLATOR

300 LB
WEIGHT CAPACITY



RHYTHM HEALTHCARE, LLC | LIFESTYLE MOBILITY AIDS
3200 Tyrone Blvd N
St. Petersburg, FL 33710

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com
V2: 6/24/21

PARTS DIAGRAM

⚠ Prior to use please read all instructions and keep manual for future use.



- | | |
|-----------------------------|----------------|
| 1. Handgrip | 5. Seat |
| 2. Brake lever | 6. Tote Bag |
| 3. Height Adjustment button | 7. Front wheel |
| 4. Adjustable Backrest | 8. Rear wheel |

UNFOLDING THE ROLLATOR

The rollator comes fully assembled. Place the rollator on its wheels on a flat surface and open by pressing down on the tubes at each side of the seat until they lock into place.



ATTACHING/REMOVING THE TOTE BAG

To remove the tote bag, grab and twist up on the round cap at the end of the tube on each side of the seat. Hold the cap to the side and slide the clip on the tote bag out of the tube. Then replace the cap into the tube.



ADJUSTING THE HANDLE HEIGHT

Press and hold the adjustment button. Simultaneously move the handgrip up or down to the desired height and release the adjustment button. The handgrip will automatically lock into the next possible height position. Make sure to adjust both sides at the same height.


⚠ The Ideal handle height is set when the top of the handle aligns with the user's wrist when standing upright.



USING THE BRAKES

To slow or stop the rollator while walking, squeeze both brake levers upward to apply the brakes. Always apply brake pressure evenly.

To lock the brakes while the rollator is stopped, push down on each brake lever until they lock into place. Release the brakes by pulling up on the brake levers.

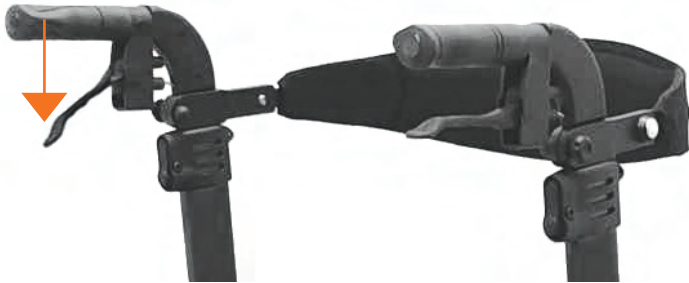

USING THE ROLLATOR AS A SEAT

Before using as a seat, the rollator should be on a level surface with the brakes in the locked position.

Lock the brakes by pushing down on the brake levers until they lock into place. Do not lean back or to the sides when seated.

FOLDING FOR TRANSPORT/STORAGE

To fold the rollator, pull up firmly on the strap located on top of the seat.

Then secure by pushing the two sides of the frame together until an audible click is heard signifying the lock is in place. Pull the handles apart to unlock.

ADJUSTING THE BACKREST

The backrest height and depth are adjustable.

To adjust the height, loosen the screw knob below the handgrip and slide the backrest up or down to the desired height and tighten the screw knob.

To adjust the depth, press and hold the silver adjustment button on the backrest and slide the backrest in or out to the desired depth. Then release the adjustment button making sure it clicks securely into place.



SAFETY PRECAUTIONS

- Before use. The user should read all instructions and familiarize themselves with the rollator.
- Ensure all components are properly and securely installed.
- Ensure the user has sufficient strength to safely operate and control the rollator and hand brakes.
- Before using the rollator as a temporary seat, ensure the brake are locked.
- Do not release brakes until user is standing.
- Do not exceed the maximum recommended weight capacity of 300 lbs evenly distributed.
- Avoid storing the rollator in direct sunlight; seat or parts exposed to sunlight will heat up and may cause injury to the skin.
- Do not place more than 10 lbs of shopping or other items into tote bag.
- ⚠️ • **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

MAINTENANCE

- ⚠️ **Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals.**

Frame

- Wipe the rollator frame with a damp cloth.
- For more stubborn dirt, use a mild detergent solution with warm water.
- Dry thoroughly before use to avoid corrosion.
- Clean the wheels using a damp brush with plastic bristles (do not use a wire brush or you may damage the rubber or painted finish).

Bag and Seat

- Regularly clean with a mild detergent and warm water, or wash with alcohol-based disinfectants if necessary.
- Dry thoroughly before use.
- Avoid contact with saltwater or pools with high levels of chlorination.

WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.

Normal wear and tear on non-durable components, such as hand grips, brake cables, wheels, and totes are not covered under this warranty. These items are subject to normal wear and need periodic replacement and/or adjustment.

For warranty service, it is recommended but not required that the product be returned to the Rhythm Healthcare, LLC LMA dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC LMA dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC LMA dealer, prior authorization will be required.

This product is identified by a serial number at the time of manufacture. Removing or defacing the serial number label may void this warranty. If you have a question about your Rhythm Healthcare, LLC LMA dealer walker or this warranty please email us at

contactus@rhythmhc.com.