



ITEM# B2200

PREMIUM BATH BENCH W/ARMS

WEIGHT CAPACITY: 300 LBS



Available colors:
Blue, White



Rhythm Healthcare
Clearwater, FL 33762

contactus@rhythmhc.com
Website: www.rhythmhc.com
(877) 843-6464

V2: 6/21/23

GENERAL EXPLANATION OF YOUR BATH BENCH




USER GUIDE

Prior to use please read all instructions.
We recommend you keep this User Guide in a safe place for future reference.

SET UP

1. Place the seat face down on a level surface.
2. Insert the four leg tubes into the frame. Depress the brass push pin and push the tube in until the pin fully engages in the frame hole [A].
3. Place the chair upright so that it is resting on all four legs and insert the arms into the frame. Depress both push pins in and push down until the pins fully engage in both frame holes. Repeat this step for the opposite arm. [B] (Note: Arms should bend back towards the chair and not away from the chair.)
4. Use the push buttons to adjust the stool to the desired seat height. Be sure the bench is level and stable before using and that all push buttons are fully engaged.

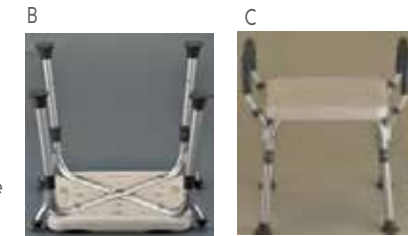

ADJUST LEG HEIGHT

1. Adjust the leg height by pushing the button and sliding the tube up or down. The height will adjust one hole at a time.
2. Ensure that all legs are adjusted to the same height.

CLEANING

Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Rhythm Healthcare product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your Rhythm Healthcare product or warranty please call your local dealer or contact us directly.