

PREMIUM BATH BENCH W/ BACK AND ARMS

300 lb Weight Capacity



Colors:
Blue and White



Rhythm Healthcare
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com
(877) 843-6464

V2: 9/6/23

GENERAL EXPLANATION OF CHAIR



USER GUIDE

! Prior to use, please read all instructions. We recommend you keep this user guide in a safe place for future reference.

SET UP

Chair Assembly

Place the seat face down on a level surface. Insert the legs into the frame. Depress brass button and push tube until it clicks into place. (A)

Insert the L-shaped back tubes into backrest tubes until brass button clicks into place. (B)


Turn the chair on side so tubing holes are facing upward. Insert (C)

Place chair upright so it's resting on all four legs and insert arms into frame. Depress both brass buttons and push down until the brass buttons click into place. Repeat for opposite arm. Arms should curve inwards toward each other.

Use the push buttons and numbered holes to adjust the stool to the desired seat height. Be sure seat is level, stable and all buttons are full engaged before using.

Adjust Leg Height

Adjust leg height by pushing button and sliding tube up or down. Height will adjust one hole at a time. Ensure all legs are adjusted to same height.

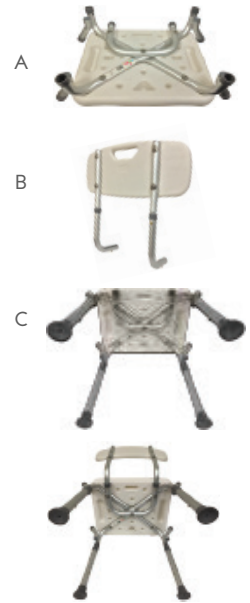

CLEANING

- !** Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.
- > Use a non-abrasive detergent or household cleaner with warm water.
 - > Rinse with warm water and wipe dry.
 - > DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Rhythm Healthcare product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your Rhythm Healthcare product or warranty please call your local dealer or contact us directly.

CLEANING