



RHYTHM
HEALTHCARE

ITEM#
B3180S/B3185E

RAISED HINGED TOILET SEAT



GENERAL EXPLANATION



SET UP

1. Remove your toilet seat.
2. Place your hinged toilet seat riser on the toilet, place your toilet seat on top of the riser, lining up the holes of the riser and the seat with the holes on the toilet bowl.
3. Put the enclosed bolts through the holes, attaching the toilet seat riser to the toilet frame with the wing nuts provided.



CLEANING

The Toilet Seat should be cleaned regularly with a mild cleaning agent and a soft cloth.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

SAFETY & MAINTENANCE

 **Inspect weekly for signs of damage or cracking. Do Not Use if damaged. Contact your Rhythm Healthcare dealer immediately.**

- › Always be sure that the seat is correctly and securely in place before using.
- › Physically challenged users should be supervised or assisted whenever using this product.
- › This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
- › If you have a style of commode that this product will not fit properly, it is recommended that you discontinue usage.
- › Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.

LIMITED LIFETIME WARRANTY

Your Rhythm Healthcare product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.



Rhythm Healthcare, LLC
Clearwater, FL 33762

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com

V2: 6/15/22