



ITEM# B4350

PREMIUM SHOWER CHAIR WITH BACK AND PADDED ARMS



Rhythm Healthcare, LLC
3200 Tyrone Blvd N
St. Petersburg, FL 33710

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com

V2: 2/24/23

yaids.com
is.com

GENERAL EXPLANATION OF YOUR SHOWER CHAIR



USER GUIDE

⚠️ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

TABLE OF CONTENTS

SPECIFICATIONS	2	This Bath Seat is constructed of heavy duty molded plastic, with an aluminium anodized frame. Comes complete with skid resistant rubber tips along with push button release removable arms and back. Great for travel and easy to store.
SET UP	3	
CLEANING	3	
WARRANTY	3	

SPECIFICATIONS

NAME	Premium Shower Chair with Back and Padded Arms
PRODUCT WEIGHT	5 lbs
SEAT HEIGHT	16" to 21"
SEAT DIMENSIONS	20" W x 12" D
UNIT DIMENSIONS	17" x 16"
WEIGHT CAP	350 lbs

SET UP

Chair Assembly

Place the seat face down on a level surface. Insert the four legs into the frame. Depress the brass button and push the tube in until the button "clicks" and locks into position [A].

Insert the L-shaped back tubes into the backrest tubes until the brass button "clicks" into locked position [B].

Turn the chair on its side so that the tubing holes are facing upward. Insert backrest into tubing holes until the brass buttons "click" and lock into position [C].

Place the chair upright so that it is resting evenly on all four legs and insert the arms into the frame. Depress both brass buttons and push down until the brass buttons "click" and lock into position. Repeat this step for the opposite arm. (Note: Arms should curve inwards towards each other)

Use the push buttons and numbered holes to adjust the stool to the desired seat height. Be sure the Bath Seat is level, stable and all push buttons fully engaged before using.

Adjust Leg Height

Adjust the leg height by pushing the button and sliding the tube up or down. The height will adjust one hole at a time.

Ensure that all legs are adjusted to the same height.



CLEANING

⚠️ Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Rhythm Healthcare product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber tips and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your product or warranty please call your local dealer or contact us directly.