

ITEM #: C400BK

## BARIATRIC OFFSET CANE WITH STRAP

WEIGHT CAPACITY: 500 LBS



### WARRANTY

Limited Lifetime warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, or improper operation and unauthorized modifications. It is recommended, but not required, that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare will determine if the product will be repaired or replaced.


Rhythm Healthcare  
Clearwater, FL 33762

contactus@rhythmhc.com  
www.rhythmhc.com

V2: 6/26/23

- ⚠ Read entire manual prior to assembly and use and save for future reference. If components are damaged or missing, contact your dealer immediately.

### GENERAL EXPLANATION

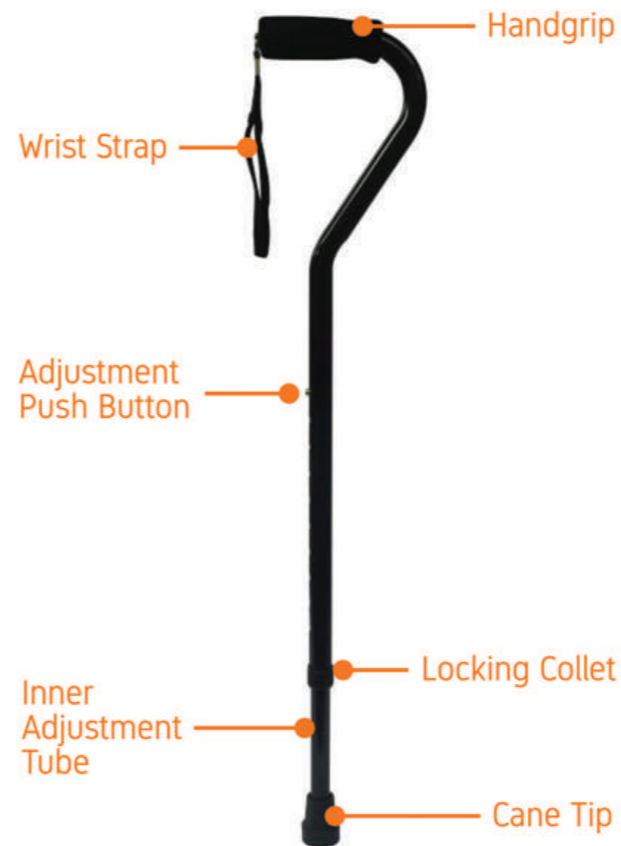

### USER INSTRUCTIONS

NOTE: Consult a healthcare professional to determine proper adjustment and usage relating to your specific needs.

#### Cane Height Adjustment

- 1 Loosen Locking Collet by turning counter-clockwise.
- 2 Press Adjustment Push Button, grab cane tip and slide inner tube to desired height until Adjustment Push Button clicks into the desired adjustment setting.
- 3 Tighten Locking Collet until secure.

### PRECAUTIONS

- › If components are damaged or missing, contact your dealer immediately. DO NOT substitute parts.
- › Before use, ensure that cane tip is not ripped, worn or missing, and is properly seated.
- › Before use, ensure that push button protrudes fully through adjustment hole and Locking Collet is secure.
- › Always use caution on wet or slippery surfaces.

### CARE & MAINTENANCE

- › Clean weekly with damp cloth. Handgrip may be cleaned with mild soap or detergent.
- › DO NOT use organic solvents, such as acetone, or abrasive cleansers.
- › Inspect weekly for wear and to ensure push button and locking collet are secure.
- › Replace cane tip as needed.
- › If any component appears worn or is not working properly, contact your dealer for repair or appropriate replacement parts.