



RHYTHM
HEALTHCARE

ITEM# H5501

DELUXE BEDSIDE ASSIST
WAVE HANDLE WITH LEGS





Prior to use, please read all instructions and inspect all parts carefully for damage prior to assembly.

H5501

Cautions and Warnings

1. Please read these instructions carefully before installing and using the bed rail.
2. This bar is not designed to support the entire weight of a person and has a weight capacity of 300 lbs.
3. Narrow beds can be tilted if user puts full body weight on the handle.
4. The adjustable bed bar is not intended for people with reduced mobility or people who are permanently bedridden.

Additional safety measures and precautions should be used in high-risk patients.


5. The bed rail must not be attached to a medical bed that can move or be lifted.
6. The mattress must be tight against the grab bar without leaving any space to prevent a person from getting trapped. This should be checked frequently, if not daily.
7. For added safety, we recommend that you install two bed rails, one on each side of the bed, to help hold the mattress in place, without leaving any space.

Warning: If the box spring is wider than the mattress, two bed rails will not work.

Assembly Instructions


1. Remove all parts from carton and dispose of plastic wrapping safely keeping away from children and infants.
2. Insert part A into part B on each side.
3. Connect part C into part D on each side using provided allen key. Do not tighten.
4. Insert part E into part D making sure that all spring buttons are in place. Tighten all screws snugly.
5. Insert parts F into part D on each side making sure spring buttons are in place.
6. Once installed, the handle height adjusts by raising or lowering the legs. Parts A and B will slide up and down part D.

*Tip : If connecting part D to part E is difficult, slightly loosen the screws connecting part C. Re-insert part E into part D making sure the spring buttons are in place and re-tighten screws.

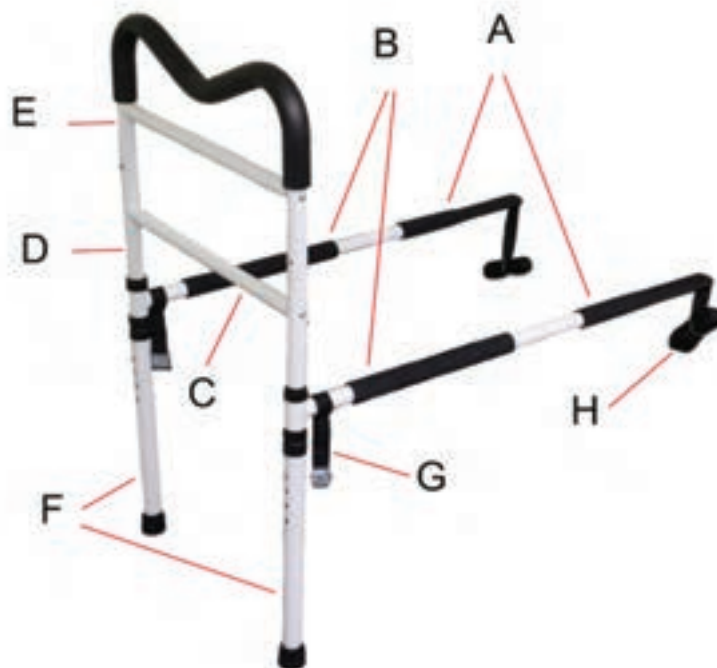

Installation Instructions

1. For easier installation, remove the mattress and place the assembled Rail on top of the boxspring.
2. Run the long webbing (part H) around the bottom of the boxspring and back to the bottom of the handle.
3. Feed the webbing through the buckle (part G) and lock the buckle to secure the strap at the desired length. Repeat on other side.
4. Replace the mattress.
5. For positioning, lie on the bed and move the bed rail to the desired position so it's easy to use the rail.
6. Making sure the webbing straps are straight and parallel, tighten them securely against the boxspring. *
7. Loop and tuck the loose end of the webbing directly below the buckle to prevent tripping hazards.

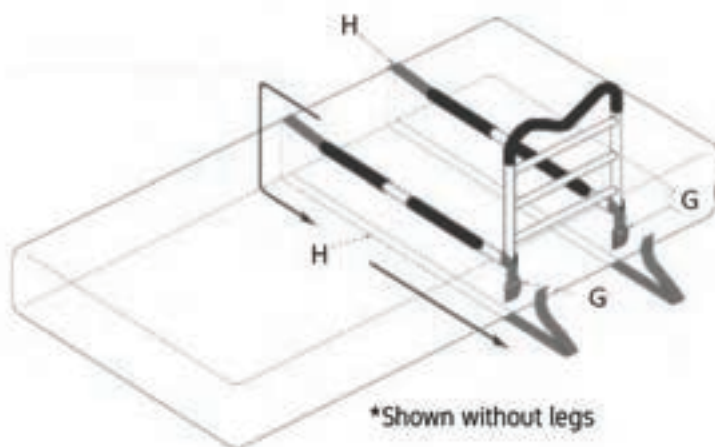

IMPORTANT : For your safety, Part F Safety Strap set provided with this bedrail **MUST** be installed as shown above

Care and Maintenance

Clean frame with soap and water, dry with towel.

Check straps and handles periodically to make sure they are not torn, worn or missing.

Replace if necessary.



Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

Rhythm Healthcare, LLC
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com
877 843 6464

v2: 9/15/23