



Item #: 820XP Venture XP - Euro Style Rollator



Available colors:



## VENTURE XP - EURO STYLE ROLLATOR

300 LB  
WEIGHT CAPACITY



RHYTHM HEALTHCARE  
Clearwater, FL 33762

Email: [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)

V2: 8/1/23

## GENERAL EXPLANATION

- ⚠ Read entire manual prior to assembly and use of this product and save this manual for future reference. If components are damaged or missing, contact your dealer immediately.



## ASSEMBLY INSTRUCTIONS

### Unfolding the Rollator

Place the rollator on a stable surface and unfold by pushing down on outer edges of the seat until the seat frame clicks into place and attach the Stabilization Bar by clipping to the Handgrip tubing.

### Adjusting Armrest Height

Insert the Handle Height Adjustment tubes into the frame and set to the desired height by sliding the tube up or down and making sure the Adjustment Buttons click into place. Be sure to set both sides at the same height. The armrest should be set at a height where your forearms are supported when standing upright. Your shoulders will "hunch" if set too high.

### Setting Handgrip Length

Adjust the length of the Handgrip by loosening the Handgrip Adjustment Knob and sliding the tube in or out to the desired length. Then tighten Handgrip Adjustment Knob until secure. Be sure to set both sides at same length.

### Adjusting the Brake Cables

Due to normal use, the brakes may need to be adjusted occasionally. On the frame above the rear wheels, use your fingers or pliers to turn the silver barrel adjuster nut a half rotation at a time until desired setting is achieved (turn clockwise to tighten or counter-clockwise to loosen). Then turn the outer locking nut clockwise until it tightens against the housing to keep the barrel adjuster nut from loosening. (See Figure 1)



Fig 1

### Attaching the Shopping Bag

Attach the side loops of the shopping bag onto the handles protruding from the front of the rollator seat. Do not place items weighing more than 10 lbs into the shopping bag.

### Folding the Rollator

Unclip the Stabilization Bar, lower the armrests and pull up on the seat handle until folded.

## OPERATING INSTRUCTIONS

- ⚠ Be sure to lock Hand Brakes when using as a seat or making adjustments to settings.

### Using the Hand Brakes

To set the Hand Brake, push forward until it clicks into locked position. To unlock, pull the brake handle toward the Hand Grip. Be sure to apply equal pressure to both brakes when stopping.

### Walking with the 820XP

Stand upright directly behind the seat and place your forearms on the Armrest Pads applying equal pressure. Place your palms on the Handgrips with your fingers around the Hand Brakes. Walk forward taking full steps. When walking downhill, apply even pressure to the Hand Brakes to control your descent.

### Using the Seat

Make sure to lock the Hand Brakes before sitting and sit with legs to rear of rollator.

## SAFETY PRECAUTIONS

- › Do not use this product without first reading and understanding the instructions in this manual.
- › Ensure that all functional or moving parts are in good working order before operation.
- › Do not apply weight disproportionately to one armrest.
- › Do not use on stairs, escalators, moving sidewalks or uneven surfaces.
- › Do not go up or down inclines greater than 15% or diagonally across inclines greater than 5%.

## MAINTENANCE

- ⚠ Professional inspection is recommended at least once a year.

- › Regularly check for damage and wear, paying special attention to the brakes and condition of the wheels. If there is any damage, stop using the product immediately and contact your authorized dealer.
- › Wipe all surfaces with a damp cloth and allow the surfaces to dry thoroughly before use.
- › Never use abrasive cleansers or scouring pads on any part of the product.
- › Clean wheels using a damp brush with plastic bristles.

## WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. Prior authorization is required for walkers returned to Rhythm Healthcare.

All Rhythm Healthcare products are identified by a serial or lot number at the time of manufacture. Removing or defacing the labels may void this warranty.

If you have questions about your rollator or this warranty, please call (877) 843-6464.

Visit [www.rhythmhc.com](http://www.rhythmhc.com) for more information about our products and accessories.