



Lifestyle
MOBILITY AIDS

COMMANDO
BY LIFESTYLE MOBILITY



AVAILABLE COLORS:



COMMANDO ROLLATOR

950BK - Commando Rollator, Black • 950BL - Commando Rollator, Blue
950WH - Commando Rollator, White

USER MANUAL - COMMANDO ROLLATOR



SAFETY NOTICE & CAUTION


- a) Consult your doctor, therapist or other qualified professional to determine if this Rollator will suit your particular needs.
- b) For proper positioning and use of your Rollator, imagine a line drawn across the back of the rear wheels. Part of one foot should always be slightly ahead of that line as you walk
- c) This rollator is for individual use only.
- d) Rollator is designed for indoor & outdoor use.
- e) Regularly check the brakes for proper operation. Pulling up on the brake handle will slow the rollator. Pushing down on the brake handle will lock the wheels. Pull up on the brake handle to release the brake lock. (refer "BRAKE USAGE" on page 2)
- f) Do NOT use as a wheelchair or transport chair.
- g) Rollator must be fully opened and locked position when being used as a walking aid.
- h) Wear proper shoes while using this rollator.
- i) Follow the instruction "OPENING/FOLDING" to fold or unfold the rollator (refer to page 2).
- j) Do NOT sit on the rollator and push yourself around using your feet to propel.
- k) Do NOT stand on the seat.
- l) Before sitting on the seat, be sure the rollator is fully opened, and the brake is in the locked position.
- m) Do NOT use on stairs or escalators.
- n) Do NOT sit on the rollator if the ground is sloped or uneven.
- o) Exercise caution when walking on sloped surfaces.
- p) Do not hang objects from the backstrap.
- q) Do not use if the rollator is not functioning properly.
- r) Maximum user weight 300lbs / 136kg .

SETUP

ATTENTION!

Ensure that all bolts are securely fastened before using this rollator!
Brakes must be in the parked /locked position before sitting on the rollator!

1. Install the brake handles

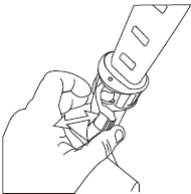

The rollator should be in the unfolded position before beginning. Holding the 'RED' release lever in the open position. Slide the brake handle into the frame. The rectangular slots on the brake handle tube should be facing inwards.

Set the desired handle height, and close the 'RED' release lever. The release lever should 'click' when locked into the closed position.


2. Install the backstrap

Slide backstrap into the backrest connector. The padded side of the backstrap should face the inside of the rollator.


Secure the backstrap into position by tightening the screw and nut.



3. Install the shopping bag

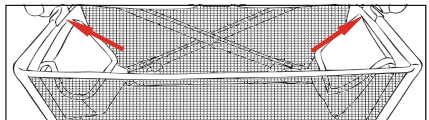

Slide the basket hooks into the brackets located under the seat at the front.

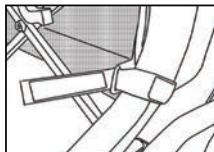

Loop the strap (velcro) around the frame and insert into the wire tab.




Ensure velcro is engaged.

**CAUTION: DO NOT OVERLOAD
SHOPPING BAG MAX LOAD 11lbs/5kg**

BRAKE USAGE




Pull up on the brake handle to engage the brake.

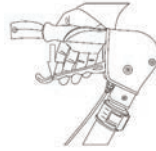

Push down on brake handle to lock the brake into PARK.

OPENING/FOLDING

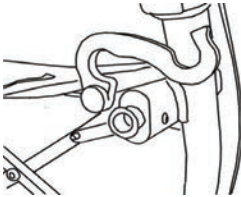

To **Unfold a rollator**, turn the frame lock clockwise to unlock the frame and then push down on the seat. Then turn the frame lock clockwise to lock the rollator in the open position.

To **Fold a rollator**, turn the frame lock counter clockwise to unlock the frame lock. Pull up and the seat handle and then fold. Turn the frame lock counter clockwise to lock the rollator in the folded position.

USAGE



Do not support your weight on the handle attempting to sit. Use the handles to stabilize your body as you carefully turn around. Keep your hand on the handles and slowly lower your weight onto the seat.

STORAGE & TRANSPORTATION


Fold your rollator for easier storage and transportation. Do not rest anything on your folded rollator when being stored or transported as damage may occur.

PRODUCT LABEL

1. Product name
2. CE-marked
3. Read the manual
4. Seat height
5. Max width
6. Max user weight/basket load
7. Indoor and outdoor use
8. Do not use as wheelchair

MAINTENANCE

- a) Clean the rollator using mild soap and water. Wipe dry.
- b) Keep wheels clean.
- c) Check screws and handle height adjustments are tight.
- d) If your rollator is not working properly. Contact your dealer.
- e) **DO NOT USE** if the rollator is not functioning properly.
- f) Use only authorized parts/accessories with your rollator.

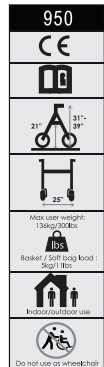

WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.


Normal wear and tear on hand grips, brake cables, wheels, and toes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended but not required that the walker be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However all Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the walker is to be returned to Lifestyle Mobility Aids, prior authorization will be required.

All Lifestyle Mobility Aids products are identified by a serial number at the time of manufacture. Removal or defacing the serial number may void this warranty. If you have a question about your Lifestyle Mobility Aids walker or this warranty please email us at contactus@lifestylemobilityaids.com.




Additional Commando Products:

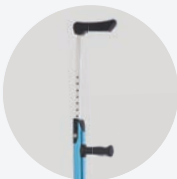

Commando Memory
Standard Crutch



Commando Memory
Forearm Crutch

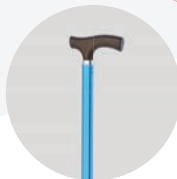

Commando Memory
Cane

Manufactured By :
LIFESTYLE MOBILITY AIDS
Email : contactus@lifestylemobilityaids.com
Website : www.lifestylemobilityaids.com

