

ITEM# B4500



500 lb Weight Capacity



Rhythm Healthcare, LLC  
3200 Tyrone Blvd N  
St. Petersburg, FL 33710

contactus@rhythmhc.com  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)  
(877) 843-6464

V1: 5/4/23

## GENERAL EXPLANATION

⚠ Read entire manual prior to assembly and use and save this manual for future reference.



## SAFETY PRECAUTIONS

- > Assistance is required for users with limited capabilities.
- > Ensure all spring pins and u-clips protrude fully through holes before use.
- > Check that chair is stable and positioned on flat surface.
- > Check leg tips for rips, crack or other wear and replace if necessary.

## CLEANING

- ⚠ Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.
- > Use a non-abrasive detergent or household cleaner with warm water.
  - > Rinse with warm water and wipe dry.
  - > DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

## SET UP

⚠ If components are damaged or missing, contact your dealer immediately.  
Do not substitute parts.

### Assembling the Cross Brace

Take the upper and lower Cross Braces Assemblies and place the flat middle areas together. Insert the Wing Bolt through the Upper Cross Brace into the Lower Cross Brace and screw on the Bolt Cap but do not tighten.

Insert each Leg into the lower tubes of the Cross Brace making sure the Leg Tips are angled outward. Set each Leg at same height using the U-Clips. Make sure the pins of the U-Clips fully protrude through the setting holes.



### Adding the Seat Assembly

While pushing in the Spring Pins on the upper tubes of the Cross Brace, insert them into the tubes of the Seat Assembly until the Spring Pins click into place. Then tighten the Wing Bolt on the Cross Brace.

### Adding the Back Rest

While pushing in the Spring Pins on the long ends of the Back Rest Attachment Tubes, insert them into the receiving brackets on the bottom side of the Seat Assembly until the Spring Pins click into place. Then push in the Spring Pins on the Back Rest Assembly tubes and insert them into the Back Rest Attachment Tubes until the Spring Pins click into place.

### Adjusting Leg Height

Remove the U-Clip, slide the leg up or down to desired height and replace the U-Clip making sure the pin protrudes fully through the setting hole. Repeat on all legs making sure to set all to same height.

## LIFETIME LIMITED WARRANTY

Your Rhythm Healthcare product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your Rhythm Healthcare product or warranty please call your local dealer or contact us directly.