

## OPERATING INSTRUCTIONS

### TO OPEN WALKER

\*Beware of pinch points when opening and folding walker.

Pull both side frames away from the front crossbrace until both locking mechanisms lock securely into place.

Ensure side frames are secure by trying to push them toward front crossbrace.

### TO FOLD WALKER

Press down on the locking mechanisms and fold the side frames toward the front crossbrace.

Before use, walker legs must be even. When using wheeled legs, wheels must be to the outside of walker.

## OPERATING INSTRUCTIONS

### INSTALLING/ADJUSTING LEG EXTENSIONS

Press the adjustment button and slide the inner tubing of the leg to the desired height making sure the adjustment button clicks securely into the adjustment hole.

Repeat this step for each leg making sure they are all set at the same height.

## MAINTENANCE

Clean frame with a damp cloth and dust weekly.

Never use abrasive cleansers or scouring pads.

If exposed to liquid, dry thoroughly as soon as possible.

If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

Check that handgrips are tight and do not twist. Do not use if handgrips are loose and contact your dealer for repair/and or replacement.

Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace tires if they become severely worn or cracking appears.

Rhythm Healthcare, LLC  
Clearwater, FL 33762

contactus@rhythmhc.com  
www.rhythmhc.com  
(877) 843-6464

v2L 10/4/23

## WARNINGS

Before use, make sure the walker is adjusted to correct height. Consult with your physician or therapist to determine the proper adjustment height and usage.

Before use, make sure walker is in open and locked position.

All legs must be set at same height to ensure the walker is properly balanced.

Always use caution when using the walker on wet, slippery or soft surfaces.

Do not use the walker on stairs or steep inclines.

Check that tips are in good condition and replace immediately if worn or missing.

When transferring from a wheelchair to a walker, be sure the wheelchair wheels are locked.

Users with limited physical strength will require assistance/supervision.

## LIMITED LIFETIME WARRANTY

Limited Lifetime warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For warranty service, it is recommended but not required that the product be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC - LMA Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC - LMA, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare, LLC - LMA will determine if the product will be repaired or replaced.

ITEM # P150X



# BARIATRIC EXTRA WIDE TWO BUTTON FOLDING WALKER

500 lb  
Weight Capacity

