

FOLDING POWDER
COATED ALUMINUM
COMMUNE 3 IN 1

300 lb
Weight Capacity



Rhythm Healthcare
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com
877 843 6464


V3: 9/13/23

PARTS DIAGRAM



ASSEMBLY INSTRUCTIONS

1. Unfold frame. Make sure sides of frame are at right angles to the rear of the frame and the open ends (of cross brace sockets) are facing center of frame.
2. With the Commode Bucket Supports toward the bottom, insert the ends of the Front Cross Brace into the Cross Brace Sockets by depressing the push pins on each end and sliding them in until the push pins click securely in place.
3. Connect the loose ends of the Commode Bucket Supports to the brackets at the rear of the frame using the Wing Bolts and nylon washer provided. Insert the Wing Bolt through the support, add the nylon washer, then thread the Wing Bolt into the attached nut and tighten until secure. Repeat for second Support.
4. Insert the Back Brace into back of frame, making sure push pins click securely into place.
5. Attach Commode Seat to rear of frame by pressing clips onto rear Cross Brace until they snap securely into place.

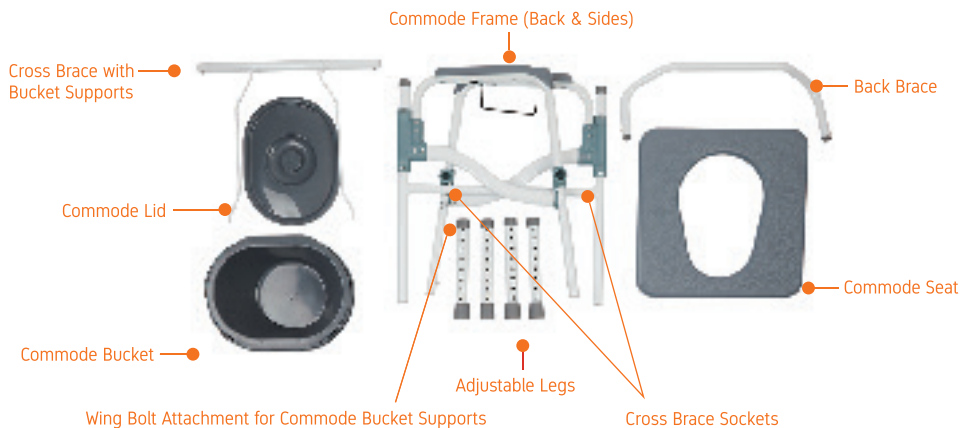

OPERATING INSTRUCTIONS

To Adjust Legs

- 1 Depress the push pin and slide the lower leg up or down to desired height until push pin clicks into place.
- 2 Repeat step 1 for each leg, ensuring they are set at same height.

Using as Bedside Commode

- 1 Lift commode seat and place bucket on support rails, making sure it is directly below the opening. Lower seat for use.
- 2 When not in use (to contain the odor), place the lid on the bucket and use handle on bucket for cleaning.

Using Commode as Toilet

- 1 Adjust height, if necessary, and remove the back brace. Then replace bucket.
- 2 Lift toilet bowl seat and place commode over toilet. Lower commode seat for use.

SAFETY PRECAUTIONS

! Rhythm Healthcare, LLC is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.

- > If you do not understand these instructions, contact your dealer or a healthcare professional for assistance.
- > If any parts are damaged or missing, contact your dealer for replacement. Do not substitute parts.
- > Ensure commode seat is securely attached to the frame and all legs are set at the same height before use.
- > Users with limited physical strength will require assistance/supervision.
- > Weight capacity is 300 lb, evenly distributed.

MAINTENANCE

- > Clean after each use with disinfectant.
- > Do NOT use organic solvents, such as acetone or abrasive cleaners.
- > Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or replacement parts.

WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended, but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization is to be required. In the event of a defect covered by this Warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced. Rhythm Healthcare, LLC products are identified by a serial or a lot number label at the time of the manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare, LLC product or this warranty, please email us at contactus@rhythmhc.com