LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For Warranty Service, it is recommended but not required, that the product be returned to the Rhythm Healthcare, LLC / Lifestyle Mobility Aids dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC / Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC / Lifestyle Mobility Aids, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare, LLC / Lifestyle Mobility Aids will determine if the product will be repaired or replaced.



Rhythm Healthcare, LLC Lifestyle Mobility Aids 3200 Tyrone Blvd N St. Petersburg, FL 33710

contactus@rhythmhc.com Website: www.rhythmhc.com (877) 843-6464

V1: 11/23/21



SHOWER STOOL WITH SWIVEL SEAT

450 lb Weight Capacity



PARTS DIAGRAM



ASSEMBLY INSTRUCTIONS

- 1. Remove stool parts from carton.
- 2. Place seat upside-down. Insert each leg into the bottom of the seat. Repeat this for the three remaining legs. Ensure that the legs are secured in place and that all four legs are at same height for stability and safety.
- 3. Turn the stool right-side-up. Place storage tray between legs while inserting the cross braces into the four slots at the bottom of the receptacle legs.
- 4. Ensure the rubber feet are in good condition and free of rips, tears and other signs of wear. All four feet must be flat on floor at all times. Stool is intended for sitting only. Do not stand on stool.
- 1. To rotate padded seat, pull up on handle until seat is free and rotate to desired position. Seat can rotate in both directions for a full 360° and locks in 12 positions.

Before user stands or sits, make sure the seat is in one of the 12 locked positions.

MAINTENANCE

To maintain the original finish, clean with mild soap and rinse with warm water.
Wipe stool dry after each use.