

## LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For Warranty Service, it is recommended but not required, that the product be returned to the Rhythm Healthcare, LLC / Lifestyle Mobility Aids dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC / Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC / Lifestyle Mobility Aids, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare, LLC / Lifestyle Mobility Aids will determine if the product will be repaired or replaced.



Rhythm Healthcare, LLC  
Lifestyle Mobility Aids  
3200 Tyrone Blvd N  
St. Petersburg, FL 33710

contactus@rhythmhc.com  
Website: www.rhythmhc.com  
(877) 843-6464

V1: 11/23/21

ITEM #  
B5250




## PREMIUM SHOWER CHAIR WITH BACK

350 lb  
Weight Capacity



## PARTS DIAGRAM

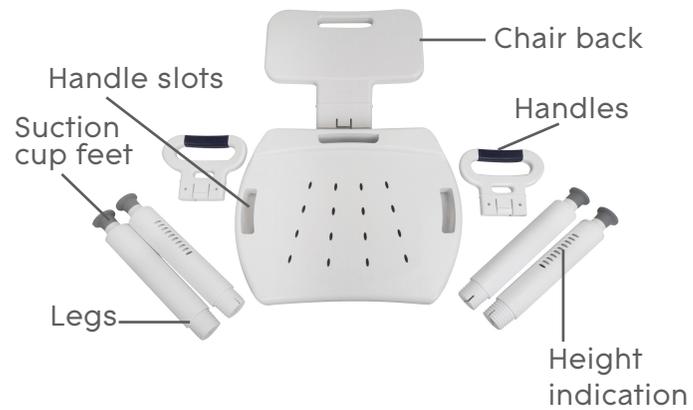

## ASSEMBLY INSTRUCTIONS

1. Remove chair parts from carton.
2. Turn seat upside down on a flat surface. Insert leg into the notch of the seat and turn right until tightened. Repeat for all four legs.
3. Turn chair upright and Insert arms into holes in side of seat until arm is securely in place.
4. Push Chair Back into slot behind seat until it is securely in place.
5. To adjust leg height, hold bottom of leg by suction cup and turn to desired height and repeat for all four legs. Always ensure all legs are at the same height for safety and stability.

## DISASSEMBLY INSTRUCTIONS

1. Press tab on chair back to release back from the seat.
2. To remove arms, pinch tabs on the arm to release.
3. Rotate the legs to remove.

## MAINTENANCE

To maintain the original finish, clean with mild soap and rinse with warm water. Wipe stool dry after each use.