

LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. Prior authorization is required for walkers returned to Rhythm Healthcare.

All Rhythm Healthcare products are identified by a serial or lot number at the time of manufacture. Removing or defacing the labels may void this warranty.

If you have questions about your rollator or this warranty, please call (877) 843-6464.

Visit www.rhythmhc.com for more information about our products and accessories.

Rhythm Healthcare
Clearwater, FL 33762



UNIVERSAL SERIES ROLLATORS

807U, 812U: 300 lb Weight Capacity
850U: 450 lb Weight Capacity



OPERATING INSTRUCTIONS

Our Universal Series Rollators have handles and legs that are height adjustable. The 807U and 812U have a weight capacity of 300lbs and the 850U has a weight capacity of 450lbs.

To assemble, unfold the rollator and insert the legs with the casters into the front of the frame. Then insert the legs with the brakes and cables to the outside of the frame.

Insert the handles into the top of the frame (with the brake cables on the outside of the frame). Clip the brake cable onto the back frame, close to the cross bar.

For proper adjustment of the handles, extend your arm toward the floor. The hand grip should be at wrist height. With your hand on the grip, your arm should be flexed 20° - 30°. Be sure to adjust both handles to the same height.

To adjust the seat height, use the push buttons on the legs to move the legs up or down to the desired height.

When using the rollator as a seat, the brakes must be in the locked position. Lock the brakes by pushing the brake handles down until they click into place and the rear wheels do not roll.

To fold for storage, lift the seat and pull up at the bar at the top of the tote. To unfold, push down on the hand grips.

MAINTENANCE

- Ensure that all hardware is secure at all times.
- Replace any broken, damaged and/or worn parts immediately.
- Clean wheels with warm water and a mild cleanser. Dry with a clean, soft cloth.
- Brakes may loosen over time and should be adjusted by a qualified dealer.
- Failure to do this may result in injury to the user.

PRECAUTIONS

- Do NOT use on stairs or curbs.
- Watch out for cords, throw rugs, toys on the floor and/or spilled liquids.
- Make sure all folding mechanisms are in the locked position when the walker is in use.
- Engage brakes when using seat.
- Do NOT use as a wheelchair.