

## HEIGHT ADJUSTABLE ROLLATOR

WEIGHT CAPACITY 300 LBS



## PARTS DIAGRAM




## ASSEMBLY INSTRUCTIONS

1. Remove all contents from carton.
2. Open frame and place up-side-down on the floor. Attach legs without brakes to the front of the walker by depressing push pin and sliding leg into the receptacle. Secure in place using the tightening knob, threading the knob in from the outside. Repeat for second leg, making sure both legs are set to the same height.
3. Attach back legs (with brakes) using the same process, making sure the wheels are on the outside of the rollator and are at the same height as the front wheels. Affix brake cable to the black portion of the leg using the "S" shaped clamp provided.
4. Insert handle tube into frame and set to desired height. Secure in place using tightening knob, threading the knob in from the outside. Repeat for second handle, making sure both are set to the same height.
5. Attach padded backrest by depressing the black thumb buttons and sliding into the black bracket. Note that back rest can be folded up when not in use.

## SAFETY PRECAUTIONS

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.
6. Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.
7. Check that the nuts and bolts are secure, on a regular basis, before use.
8. Do NOT use on stairs or curbs.
9. Watch out for cords, throw rugs, toys on the floor and/or spilled liquids.

## LOOP LOCK OPERATION

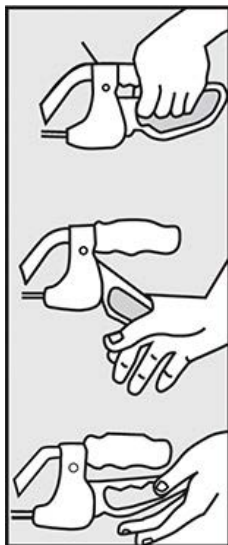

Always engage and disengage wheel locks on both sides simultaneously.

1. To slow down or stop—Pull up slowly on the loop locks with four fingers until unit slows down or stops.
2. To Temporarily Lock Wheels in Position—Press-down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks—Push up on both loop locks with palms of hand to original position.

## HEIGHT ADJUSTMENT

The height of the rollator can be adjusted in two ways to accommodate many different users.

## HANDLE HEIGHT ADJUSTMENT

1. Remove tightening knob from handle.
2. Set handle to desired height.
3. Replace tightening knob into handle, making sure to insert knob from the outside.
4. Repeat for second handle, making sure both handles are set to the same height setting.

## LEG HEIGHT ADJUSTMENT

1. Remove tightening knob from leg.
2. Depress push pin and slide leg to desired height.
3. Replace tightening knob into leg, making sure to insert knob from the outside.
4. Repeat for second, third and fourth legs, making sure all are set to the same height setting.

## MAINTENANCE

- Ensure that all hardware is secure at all times.
- Replace any broken, damaged and/or worn parts immediately.
- Clean wheels with warm water and a mild cleanser. Dry with a clean, soft cloth.
- Brakes may loosen over time and should be adjusted by a qualified dealer. Failure to do this may result in injury to the user.

## LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels and tates are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended, but not required, that the walker be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. Prior authorization is required for walkers returned to Rhythm Healthcare.

All Rhythm Healthcare products are identified by a lot number at the time of manufacture. Removing or defacing the lot number may void this warranty.

If you have questions about your rollator or this warranty, please call (877) 843-6464.

Visit [www.rhythmhc.com](http://www.rhythmhc.com) for more information about our products and accessories.



W: [www.rhythmhc.com](http://www.rhythmhc.com)  
E: [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)

V4: 9/13/24