



970BK Rimor LT  
(Black)



## RIMOR LT ROLLATOR

300 LB  
WEIGHT CAPACITY



RHYTHM HEALTHCARE  
Clearwater, FL 33760

Email: [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)  
V1: 5/30/23

## PARTS DIAGRAM

⚠ Prior to use please read all instructions and keep manual for future use.



- |                             |                |
|-----------------------------|----------------|
| 1. Handgrip                 | 5. Seat        |
| 2. Brake lever              | 6. Tote Bag    |
| 3. Height Adjustment button | 7. Front wheel |
| 4. Adjustable Backrest      | 8. Rear wheel  |

## UNFOLDING THE ROLLATOR

The rollator comes fully assembled. Place the rollator on its wheels on a flat surface and open by first releasing the orange clasp. Then push down on the seat until it is fully open.



2

## ATTACHING/REMOVING THE TOTE BAG


To remove or place the tote bag, the rollator needs to be folded up halfway first to allow space to adjust. To remove the tote bag, grab one corner and slide it up and out. To replace, line up cartridge into slot until it slides into place.



## ADJUSTING THE HANDLE HEIGHT

Turn the adjustment knob to the left to loosen and push up or down to desired height.

The Ideal handle height is set when the top of the handle aligns with the user's wrist when standing upright.

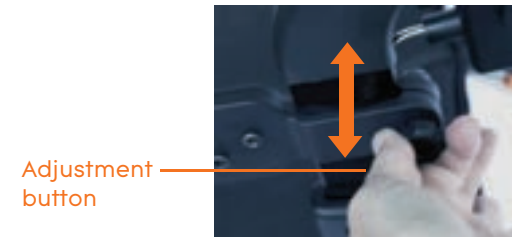


3

## USING THE BRAKES

To slow or stop the rollator while walking, squeeze both brake levers upward to apply the brakes. Always apply brake pressure evenly.

To lock the brakes while the rollator is stopped, push down on each brake lever until they lock into place. Release the brakes by pulling up on the brake levers.

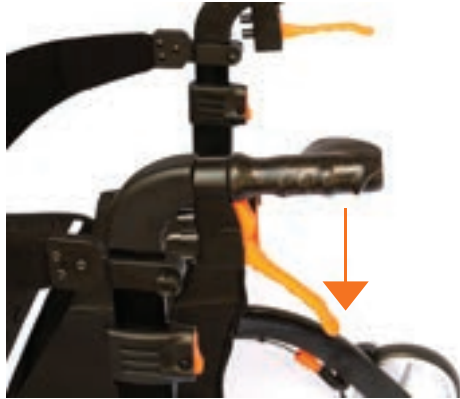

## USING THE ROLLATOR AS A SEAT

Before using as a seat, the rollator should be on a level surface with the brakes in the locked position.

Lock the brakes by pushing down on the brake levers until they lock into place. Do not lean back or to the sides when seated.



## FOLDING FOR TRANSPORT/STORAGE

To fold the rollator, pull up firmly on the strap located on top of the seat.

Then secure by pushing the two sides of the frame together until an audible click is heard signifying the lock is in place. Secure the orange clasp.



## SAFETY PRECAUTIONS

- Before use. The user should read all instructions and familiarize themselves with the rollator.
- Ensure all components are properly and securely installed.
- Ensure the user has sufficient strength to safely operate and control the rollator and hand brakes.
- Before using the rollator as a temporary seat, ensure the brake are locked.
- Do not release brakes until user is standing.
- Do not exceed the maximum recommended weight capacity of 300 lbs evenly distributed.
- Avoid storing the rollator in direct sunlight; seat or parts exposed to sunlight will heat up and may cause injury to the skin.
- Do not place more than 10 lbs of shopping or other items into tote bag.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

## MAINTENANCE

- ⚠ **Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals.**

### Frame

- Wipe the rollator frame with a damp cloth.
- For more stubborn dirt, use a mild detergent solution with warm water.
- Dry thoroughly before use to avoid corrosion.
- Clean the wheels using a damp brush with plastic bristles (do not use a wire brush or you may damage the rubber or painted finish).

### Bag and Seat

- Regularly clean with a mild detergent and warm water, or wash with alcohol-based disinfectants if necessary.
- Dry thoroughly before use.
- Avoid contact with saltwater or pools with high levels of chlorination.

## WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.

Normal wear and tear on non-durable components, such as hand grips, brake cables, wheels, and totes are not covered under this warranty. These items are subject to normal wear and need periodic replacement and/or adjustment.

For warranty service, it is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare dealer, prior authorization will be required.

This product is identified by a serial number at the time of manufacture. Removing or defacing the serial number label may void this warranty. If you have a question about your Rhythm Healthcare dealer walker or this warranty please email us at [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com).