

## CRUTCH HOLDER - 9AC06

**included:** 1x screw, 1x nut, 1x holder, part 1 (for bottom), 1x holder part 2 (for top)

**TOOL FREE**  
ASSEMBLY



Figure 1



Figure 2

### SET UP

1. Clip part 1 (clip with holes for canes around frame at the desired position on the bottom. May need to use pressure until it clicks into place.
2. Once in position, place the screw in the hole until it is all the way through both holes. See figure 2.
3. Place bolt on the under side and tighten into place on screw.
4. Hold both screw and nut and turn screw knob to the right until it is tightened fully.
5. Take part 2 and clip it into place on the frame going up to handle bars. May need to use pressure until it clicks into place. Choose desired spot which will hold part 2.

One Year Limited warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For Warranty Service, it is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare will determine if the product will be repaired or replaced.

Rhythm Healthcare  
Clearwater, FL 33762

contactus@rhythmhc.com  
www.rhythmhc.com