

ITEM# B3050



TOILET SAFETY FRAME W/ PADDED ARMS



USER MANUAL - RAISED TOILET SEAT WITH HANDLES

GENERAL EXPLANATION OF YOUR RAISED TOILET SEAT



USER GUIDE

⚠ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

SET UP

Seat Assembly


Remove safety frame from packaging and check for cracks and damages. If you see any, or suspect a fault, do not assemble the product; contact your supplier immediately.

Mount the plate [A] on the toilet bowl using the existing screws; unscrew the toilet seat, place the plate [A] and screw them back together.

Slide both tubes [C] into the frame [A] by depressing push buttons and sliding them into the frame. Beware risk of injury! Make sure that push buttons "snap" correctly through holes and that the frame is stable.

Slide both tubes [B] into the tubes [C], small stickers with the (B) letter must be positioned face to face. Screw both elements on each side together using the knurled screw and nut.

Position the armrests on the toilet bowl and fold down the lid to stabilize the toilet safety frame. Check the stability before using the safety frame.

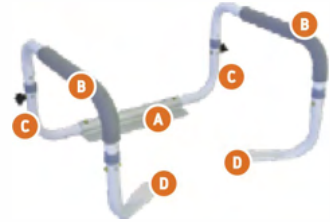

CLEANING

! The Toilet Safety Frame has to be cleaned regularly with a mild cleaning agent and a soft cloth.

- , Use a non-abrasive detergent or household cleaner with warm water.
- , Rinse with warm water and wipe dry.
- , DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

SAFETY & MAINTENANCE

General Safety Precautions

- , Always be sure that the seat is correctly and securely in place before using.
- , Physically challenged users should be supervised or assisted whenever using this product.
- , This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
- , If you have a style of commode that this product will not fit properly, it is recommended that you discontinue usage.
- , Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.

LIFETIME LIMITED WARRANTY

Your Rhythm Healthcare, LLC product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Rhythm Healthcare, LLC
Lifestyle Mobility Aids
3200 Tyrone Blvd N
St. Petersburg, FL 33710

contactus@rhythmhc.com
Website: www.rhythmhc.com
(877) 843-6464

V2: 11/30/21