

STANDARD RAISED TOILET SEAT

Weight Capacity: 300 lbs



Rhythm Healthcare, LLC
Clearwater, FL 33762

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com

V3: 6/20/23

GENERAL EXPLANATION OF YOUR RAISED TOILET SEAT



USER GUIDE

⚠️ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

TABLE OF CONTENTS

FEATURES	2
SETUP	3
CLEANING	3
WARRANTY	3

FEATURES

- Intended for individuals who have difficulty sitting down or getting up from the toilet
- All composite design prevents germ build up and cleans easily
- Fits most standard and elongated commodes
- Adds 4" to height of toilet seat
- Lightweight and portable
- Simple, tool-free installation

SET UP

Seat Assembly

- 1 Lift toilet seat and toilet lid [A].
- 2 Place Raised Toilet Seat on brim of toilet [B].
- 3 Ensure that the Raised Toilet Seat is level and stable before use [C].



CLEANING



- Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.
- > Use a non-abrasive detergent or household cleaner with warm water.
 - > Rinse with warm water and wipe dry.
 - > DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

WARRANTY

Limited Lifetime warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For Warranty Service, It is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare Dealers are qualified to assist you in obtaining warranty service.

If the product is to be returned to Rhythm Healthcare, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare will determine if the product will be repaired or replaced.

This Rhythm Healthcare product is identified by a serial or lot number label at the time of manufacture. Removing or defacing the label may void this warranty.

If you have questions about your Rhythm Healthcare product or this warranty, please contact us at contactus@rhythmhc.com.