

LIMITED LIFETIME WARRANTY

Limited Lifetime warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For warranty service, it is recommended but not required, that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization will be required.

In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.

ITEM#: B3125



RAISED TOILET SEAT

300 lb
Weight Capacity



SET UP

Seat Assembly

- 1 Lift toilet seat and toilet lid.



- 2 Place Raised Toilet Seat on brim of toilet.



- 3 Ensure that the Raised Toilet Seat is level and stable before use.



CARE & MAINTENANCE

- ⚠ DO not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

> Use a non-abrasive detergent or household cleaner with warm water.

> Rinse with warm water and wipe dry.

> DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.



Rhythm Healthcare, LLC | Lifestyle Mobility Aids
3200 Tyrone Blvd N
St. Petersburg, FL 33710

(877) 843-6464
contactus@rhythmhc.com
www.rhythmhc.com