



ITEM# B3150

RAISED TOILET SEAT W/ REMOVABLE ARMS



Rhythm Healthcare, LLC
3200 Tyrone Blvd N
St. Petersburg, FL 33710

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com

V2: 11/11/21

GENERAL EXPLANATION



ASSEMBLY INSTRUCTIONS

Seat Assembly

- 1 Remove the Seat Riser and Armrests (2) from packaging.
- 2 Match each Armrest to the corresponding Armrest Socket and push into each Socket until push pins click into place.
- 3 Pull upward on each Armrest to ensure they are securely attached before using.

Installation

- 1 Lift the toilet seat and lid. Then hook the flange at the back of the Seat Riser under the rear rim of the toilet bowl and lower the front of the Seat Riser so the adjustable bracket is inside the front of the toilet rim.
- 2 Turn the adjustment knob counterclockwise until the bracket rests firmly against the inside of the toilet rim.
- 3 When installed correctly, the Seat Riser should not move.

SAFETY PRECAUTIONS

- ⚠ Rhythm Healthcare is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.
- > Inspect weekly for signs of damage or cracking. If damaged, do not use and contact your Rhythm Healthcare Dealer immediately.
- > Always check that Seat Riser and Armrests are secure before each use.
- > The weight capacity for this product is 300 lbs evenly distributed.
- > Users should always make sure weight is evenly distributed on seat and apply even pressure to each armrest when sitting or standing up from seat.
- > Do not lean excessively to either side, forward or backward.
- > Users with limited physical strength should be supervised or assisted.

MAINTENANCE

- ⚠ Do not use abrasive cleansers or cloth to clean this product.

- > Use a non-abrasive detergent or household cleaner with warm water and a soft cloth to clean.
- > After cleaning, rinse thoroughly with warm water and dry with a soft cloth.

WARRANTY

Limited Lifetime warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For Warranty Service, It is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare Dealers are qualified to assist you in obtaining warranty service.

If the product is to be returned to Rhythm Healthcare, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare will determine if the product will be repaired or replaced.

This Rhythm Healthcare product is identified by a serial or lot number label at the time of manufacture. Removing or defacing the label may void this warranty.

If you have questions about your Rhythm Healthcare product or this warranty, please contact us at contactus@rhythmhmc.com.