

# BARIATRIC FOLDING COMMmode

550 lb  
Weight Capacity



Rhythm Healthcare  
Clearwater, FL 33762

[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
[www.rhythmhc.com](http://www.rhythmhc.com)  
877 843 6464

V2: 8/20/23

## DIAGRAM



- ⚠ If Commode Seat has been exposed to cold, allow to reach room temperature before assembly to prevent cracking.
- ⚠ Read entire manual prior to assembly and save for future use.

## ASSEMBLY

- 1 Unfold frame (A) making sure sides of frame are at right angles to the rear of the frame and the open ends of the Cross Brace Sockets are facing the center of the frame (B).

A



B



- 2 Swing the bucket support frame down and insert the ends of the cross brace into the cross brace sockets on each side frame by depressing the push pins and pushing until they click securely into place (C).

C



- 3 Insert the back brace into the back of the frame making sure push pins click securely into place.

- 4 Attach commode seat by resting the groove at the rear of commode seat on the rear cross brace and aligning the grooves alongside the commode bucket with the side rails on the bucket support frame. Then lower the front of the commode seat onto the front cross brace and push down until it

D



- 5 Insert the commode bucket from the rear.

### ADJUST LEG HEIGHT

Adjust the leg height by pushing the button and sliding the tube up or down. The height will adjust one hole at a time. Ensure that all legs are adjusted to the same height.

## OPERATING INSTRUCTIONS

### USING AS A BEDSIDE COMMODE

Lift commode seat and place commode bucket on support rails. Make sure it is directly below the opening. Then lower commode seat for use. Place the lid on the commode bucket when not in use to contain odor and use the handle on commode bucket to remove for cleaning.

### USING COMMODE OVER TOILET

- 1 Adjust height, if necessary, and remove the back brace. Then replace the commode bucket with the splash guard.
- 2 Lift the toilet bowl seat and place the commode over the toilet centering the splash guard directly over the toilet bowl. Then lower the commode seat for use.

## SAFETY PRECAUTIONS

- ⚠ Rhythm Healthcare, LLC is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.
  - > If you don't understand these instructions, contact your dealer or a health care provider for assistance.
  - > If any parts are missing or damaged, contact your dealer for replacement. Do not substitute parts.
  - > Ensure Commode Seat is securely attached to frame and all legs are set at the same height before use. Users with limited physical strength will require assistance/supervision.

## MAINTENANCE

Clean after each use with disinfectant.

- > Do NOT use organic solvents, such as acetone or abrasive cleansers.
- > Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

## WARRANTY

Limited Lifetime warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

For warranty Service, it is recommended but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.

Rhythm Healthcare, LLC commodes are identified by a lot number at the time of manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare, LLC product or this warranty, please email us at [contactus@rhythmhmc.com](mailto:contactus@rhythmhmc.com).