

## DELUXE BARIATRIC DROP ARM COMMODE

1000 lb  
Weight Capacity



Rhythm Healthcare  
Clearwater, FL 33762

[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
[www.rhythmhc.com](http://www.rhythmhc.com)  
877 843 6464


V3: 9/13/23

## DIAGRAM



## ASSEMBLY

- ⚠ If seat has been exposed to cold, allow to reach room temperature before assembly to prevent cracking.
- ⚠ Read entire manual prior to assembly and save for future reference.

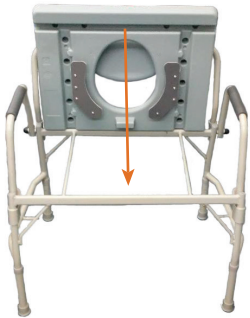

- 1 Insert Backrest into back of frame making sure adjustment buttons click into place.
- 2 Rest back edge of Commode Seat on back of frame, flip down onto front rail and then press until Commode Seat snaps securely into place.
- 3 Lift Drop arms until they click into place securely.
- 4 Slide Commode Bucket into guides from front.

## ADJUSTING LEG HEIGHT

- 1 Press the adjustment button and slide leg up or down to desired height until adjustment button clicks securely into adjustment hole.
- 2 Repeat step 1 for each leg making sure they are all set to same height.

## ADJUSTING DROP ARM

- 1 Push Drop Arm Release toward front of chair to lower for easy lateral transfer.
- 2 Reset Drop Arm by raising it until Drop Arm Release clicks back into place

## SAFETY PRECAUTIONS

Rhythm Healthcare, LLC is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.

- > If you do not understand these instructions, contact your dealer or a healthcare professional for assistance.
- > If any parts are damaged or missing, contact your dealer for replacement. Do not substitute parts.
- > Ensure commode seat is securely attached to the frame and all legs are set at the same height before use.
- > Users with limited physical strength will require assistance/supervision.

## MAINTENANCE

- > Clean after each use with disinfectant.
- > Do NOT use organic solvents, such as acetone or abrasive cleaners.
- > Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or replacement parts.

## WARRANTY

### Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended, but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization is to be required. In the event of a defect covered by this Warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced. Rhythm Healthcare, LLC products are identified by a serial or a lot number label at the time of the manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare, LLC product or this warranty, please email us at [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)