

## DROP ARM COMMODE

300 lb  
Weight Capacity



Rhythm Healthcare  
Clearwater, FL 33762

[contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
[www.rhythmhc.com](http://www.rhythmhc.com)  
877 843 6464

v3: 9/13/23

## PARTS DIAGRAM



## ASSEMBLY INSTRUCTIONS

**⚠️ Read entire manual prior to assembly and save for future use.**

### TO SET UP COMMODOE FRAME

- 1 Remove contents from packaging and if any parts are missing or damaged, contact your dealer.
- 2 Lay one Side Frame on a flat surface with the sockets for the Bucket Frame facing up.
- 3 Depress the push pins and Insert the Bucket Frame into the sockets of the Side Frame until the push pins click securely into place. The rear of the Bucket Frame has two tabs that should face up when inserted. The Drop Arm Releases are at the front of the Side Frame.
- 4 Repeat on the other side with the opposite Side Frame .
- 5 Insert the Backrest into back of frame making sure push pins click securely into place.
- 6 Align the clips at the rear of the Commode Seat with the center and press down until the Commode Seat snaps onto the frame. Repeat process with the Commode Lid.

### TO INSTALL AND ADJUST LEGS

- 1 Depress the push pin and slide the Lower Leg up or down to desired height until push pin clicks securely into adjustment hole.
- 2 Repeat step 1 for each Leg making sure they are all set to the same height.

### TO OPERATE DROP ARMS

- 1 Push Drop Arm Release toward front of commode to lower for easy lateral transfer.
- 2 Reset Drop Arm by raising arm until Drop Arm Release clicks back into place.

## OPERATING INSTRUCTIONS

### USING AS BEDSIDE COMMODOE

- 1 Lift Commode Seat and place Commode Bucket on Support Rails of the Bucket Frame. Then lower Commode Seat for use.
- 2 Place the lid on the Commode Bucket when not in use to contain odor and use the handle on Commode Bucket to remove for cleaning.

### USING COMMODOE OVER TOILET

- 1 Adjust height, if necessary, and remove the Backrest. Then replace the Commode Bucket with the Splash Guard.
- 2 Lift the toilet bowl seat and place the Commode over the toilet centering the Splash Guard directly over the toilet bowl. Then lower Commode Seat for use.

## SAFETY PRECAUTIONS

- ⚠️** Rhythm Healthcare is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.
- › If you do not understand these instructions, contact your dealer or a health care professional for assistance.
  - › If any parts are missing or damaged, contact your dealer for replacement. Do not substitute parts.
  - › Ensure Commode Seat is securely attached to frame and all legs are set at the same height before use.
  - › Users with limited physical strength will require assistance/supervision.

## MAINTENANCE

- > Clean after each use with disinfectant.
- > Do NOT use organic solvents, such as acetone or abrasive cleaners.
- > Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or replacement parts.

## WARRANTY

### Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended, but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization is to be required. In the event of a defect covered by this Warranty, Rhythm Healthcare will determine if the product will be repaired or replaced. Rhythm Healthcare products are identified by a serial or a lot number label at the time of the manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare product or this warranty, please email us at [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)