

LIMITED LIFETIME WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation and/or unauthorized modifications.

It is recommended, but not required, that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare will determine if it is to be repaired or replaced.

For questions, please email us at contactus@rhythmhc.com.

Visit www.rhythmhc.com for more information about our products and accessories.



Rhythm Healthcare, LLC
3200 Tyrone Blvd N
St. Petersburg, FL 33710

(877) 843-6464
contactus@rhythmhc.com
www.rhythmhc.com

ITEM#: C102




ALUMINUM ADJUSTABLE OFFSET CANE



300 lb
Weight
Capacity

GENERAL EXPLANATION

Read entire manual prior to assembly and save for future reference. If components are damaged or missing, contact your dealer immediately.

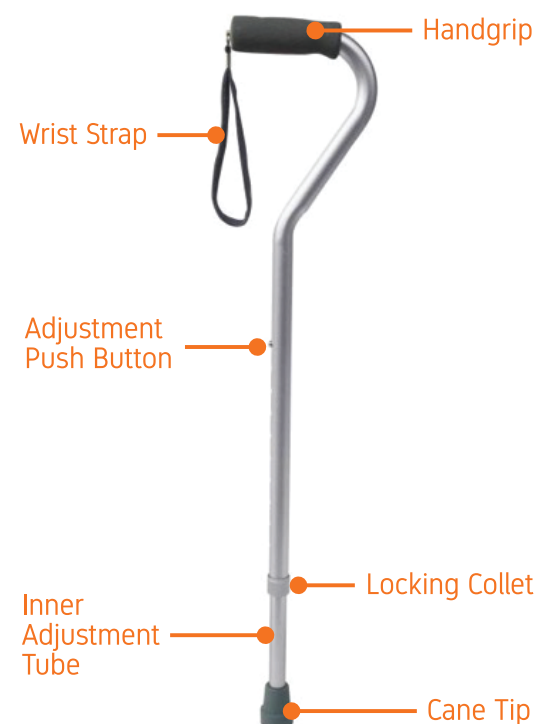

CARE & MAINTENANCE

- Clean weekly with damp cloth. Handgrip may be cleaned with mild soap or detergent.
- Do NOT use organic solvents such as acetone or abrasive cleansers.
- Inspect weekly for wear and to ensure push button and locking collet are secure.
- Replace cane tip as needed.
- If any component appears worn or is not working properly, contact your dealer for repair or appropriate replacement parts.

USER INSTRUCTIONS

Note: Consult a healthcare professional to determine proper adjustment and usage for your specific needs.

Care Height Adjustment:

1. Loosen locking collet by turning counter-clockwise.
2. Press adjustment push button, grab cane tip and slide inner tube to desired height until adjustment push button clicks into the desired setting.
3. Tighten locking collet until secure.

PRECAUTIONS

- If components are damaged or missing, contact your dealer immediately. Do NOT substitute parts.
- Before use, ensure that cane tips are not ripped, worn or missing, and are properly seated.
- Before use, ensure that push button protrudes fully through adjustment hole and screw knob is secure.
- Always use caution on wet or slippery surfaces.