

SAFETY PRECAUTIONS

Improper use of this product may result in personal injury and/or property damage. Rhythm Healthcare - LMA is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product. If you discover a problem, contact your Rhythm Healthcare - LMA Dealer for assistance.

This product is designed as a portable surface for reading, eating and keeping small personal items within reach for INDOOR use only. The weight capacity is 40 lbs evenly distributed. Do not exceed this limit. Do not lean on the suspended end of the table top as this may cause the table to tip. Do not use as a walker or as an aid when sitting down or standing up.

MAINTENANCE

Clean, as necessary, with a soft cloth using a mild detergent and water. Rinse and dry thoroughly before use. Do not use organic solvents, such as acetone or abrasive cleansers. Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

WARRANTY

Limited Lifetime warranty is provided against any factory defects in material or workmanship of the frame. One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For warranty service, it is recommended but not required that the product be returned to the Rhythm Healthcare - LMA dealer through whom it was originally purchased. However, all Rhythm Healthcare - LMA Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare - LMA, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare - LMA will determine if the product will be repaired or replaced.

Rhythm Healthcare - LMA products are identified by a serial or lot number label at the time of manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare - LMA product or this warranty, please contact us at contactus@rhythmhc.com.

Rhythm Healthcare, LLC
Lifestyle Mobility Aids
3200 Tyrone Blvd N
St. Petersburg, FL 33710

contactus@rhythmhc.com
Website: www.rhythmhc.com
(877) 843-6464
v1: 10/28/21



ITEM# H800-1-001

ADJUSTABLE OVERBED TABLE

40 lb Weight Capacity




GENERAL EXPLANATION



! Prior to assembly please read all instructions and save for future use. If you do not understand any part of these instructions, contact a health care professional or authorized dealer for assistance.

! If any components are damaged or missing, contact your authorized dealer immediately.

- | | |
|--|---|
| 1. Table Top | 9. Casters, Non-Locking (2 each) |
| 2. Column Assembly (Release Rod not shown) | 10. Bolt Sleeves (2 each) |
| 3. Adjustment Lever | 11. Bolt Plate |
| 4. 1/4 - 20 x 2 1/8" Bolt & Nut (1 each) | 12. 3/8 - 15 x 2" Hex Bolts (2 each) |
| 5. Mounting Bracket | 13. Black Screw (DO NOT REMOVE until assembled) |
| 6. Domed Phillips Head Bolts & Lock Washers (2 each) | 14. Chrome Screw A (DO NOT REMOVE) |
| 7. Base | 15. Chrome Screw B (To replace Black Screw after assembled) |
| 8. Casters, Locking (2 each) | |

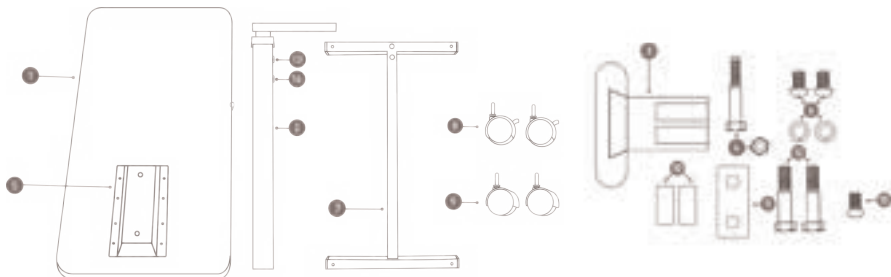

ASSEMBLY INSTRUCTIONS

! DO NOT remove the Black Screw from Column Assembly until Table is fully assembled. This product utilizes a high tension, spring loaded mechanism and could raise suddenly causing personal injury. DO NOT over tighten or use power tools to assemble.

- Place the Table Top (1) on a level surface so the Mounting Bracket (5) faces up.
- Check the Column Assembly (2) to ensure the Release Rod (not shown) is completely inside and not protruding.
- With the curved side toward table top, insert Adjustment Lever (3) into opening at top of Column Assembly (2) making sure flat side of Adjustment Lever (3) rests on release Rod. (Fig 1)
- While holding Adjustment Lever, insert short end of Column Assembly (2) into Mounting Bracket (5) and insert the two Domed Phillips Head Bolts and Lock Washers (6) but do not tighten. (Fig. 2)
- Align holes of Adjustment Lever (3), Column Assembly (2) and Mounting Bracket (5) and secure with 1/4 - 20 x 2 1/8" Bolt and Nut (4). (Fig. 1) Then, tighten the two Domed Phillips Head Screws (6) at top of Mounting Bracket (5).
- Take Base (7) with caster holes facing away from Table Top and place end with mounting holes on Column Assembly (2). Insert two large hex bolts (12) through Bolt Plate (11), through Bolt Sleeves (10), through Column Assembly (2). Insert two large hex bolts (12) through Bolt Plate (11), through Bolt Sleeves (10), through the Base (7) and into Column Assembly (2). Tighten with wrench. (Fig 2)

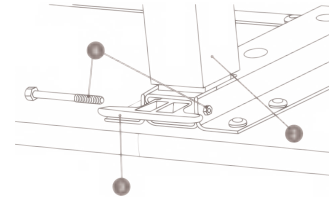


Figure 1

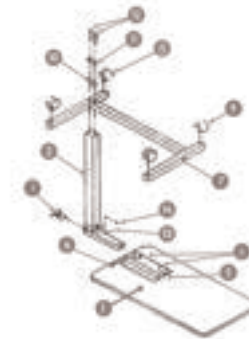


Figure 2

- Insert the Locking Casters (8) into the Base near the Column Assembly (2) and insert the Non-Locking Casters (9) into the opposite end of the Base (7). Be sure to press the casters in until they lock into place.
- Turn the table upright placing the Casters on the floor. While pressing down on the Table Top, remove the Black Screw in the Column Assembly (2) and replace it with Chrome Screw B (15) included in the hardware package.

DO NOT REMOVE Chrome Screw A (14) marked "Do Not Remove".

OPERATING INSTRUCTIONS

To Raise Table Top

Apply light upward pressure anywhere on the table top and stop when the desired height is reached.

To Lower Table Top

Pull the Adjustment Lever upward and push down on the table top at the Column end. Do not put pressure on the opposite end. When the Adjustment Lever is released, the Table Top will lock into place.

To Lock and Unlock Casters

The Casters on the Column end can be locked by stepping down on the tabs. To unlock, pull up on the tabs.