

SOFTCAIRE APP WITH PAD AND FLAPS

WEIGHT CAPACITY: 280 LBS



Alternating Pressure Pump & Bubble Pad

User Manual

Standard Features

- ◆ Simple to operate.
- ◆ Portable and lightweight.
- ◆ Pad consists of 130 bubble cells.

Operating Instructions

1. Position pump on horizontal surface or hang it with the hook.
2. Open mattress on the bed. Secure it with end flaps or straps. Air tubes must be at foot of bed.
3. Connect air feed tubes to air outlet on the mattress and pump, ensuring that the tubes are not kinked or twisted. Allow space for bed rails to drop freely.
4. Plug the pump into a suitable electrical power source and switch on. The switch indicator should now illuminate.
5. Allow 20 minutes for mattress to inflate fully.
6. Cover with a sheet and tuck loosely.
7. Please use pressure adjustable knob to give maximum patient comfort.

Technical Specifications

Pump	Mattress
Power supply: 120V/60Hz	Size: 110" L x 35" W x 2.6" H
Air output: 5 liter/min.	130 bubble cells
Pressure range: 45 mmHg – 110 mmHg	Without vents and with end flaps
Cycle time: 10 min.	Material: PVC (EN-71, low toxic)
Power consumption: 7W	Color: Beige
Fuse: 1A	Repair kit included
Size: 10.2" L x 4.3" W x 3.75" H	
Weight: 2lbs	

Cleaning Guidelines

- ◆ Pad can be cleaned with disinfectant solution.
- ◆ Do not immerse the pump in water.
- ◆ Do not heat or steam in autoclave.

WARRANTY:

Your Rhythm product is warranted to be free of defects in materials and workmanship for one (1) years of the original consumer purchaser.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and need periodic replacement.

For warranty service, it is recommended but not required that the product be returned to the Rhythm Healthcare dealer through whom it was originally purchased. However, all Rhythm Healthcare dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, prior authorization will be required.

This product is identified by a serial number at the time of manufacture. Removing or defacing the serial number label may void this warranty. If you have a question about your Rhythm Healthcare product or this warranty please email us at contactus@rhythmhc.com



Rhythm Healthcare
Clearwater, FL 33762

Email: contactus@rhythmhc.com
Website: www.rhythmhc.com

V2: 7/25/23