#### Guide to LM5BA Filters

**2**RHYTHM

# RHYTHM HEALTHCARE

# STATIONARY OXYGEN CONCENTRATOR FILTERS

The LM5BA 5L Stationary Oxygen Concentrator has 3 filters. For optimal performance and longevity of the unit and to preserve the Manufacturer's Warranty, Rhythm Healthcare provides the following guidance for filter cleaning and replacement.

## AIR INTAKE (SPONGE) FILTER

#### SKU LM5BA-ITF

The Sponge Filter is located on the back of the unit. It should be removed and cleaned on a weekly basis using warm soapy water. Use caution to <u>COMPLETELY</u> dry the filter before replacing. The Sponge Filter should also be replaced between patients. The filter should be replaced if it is torn or damaged.

# INLET FILTER (HEPA FILTER)

#### SKU LM5BA-ILF

The Inlet Filter (HEPA) is located on the back of the unit. Inspect the HEPA Filter in between patients and replace the filter if needed, or every 3 years.

## **BACTERIAL FILTER**

#### SKU LM5BA-BF

The Bacterial Filter is located inside the LM5BA Concentrator. This filter should be changed during each compressor servicing.