

## OXYGEN CART HOLDER INSTRUCTIONS (ITEM# OTHWM01)

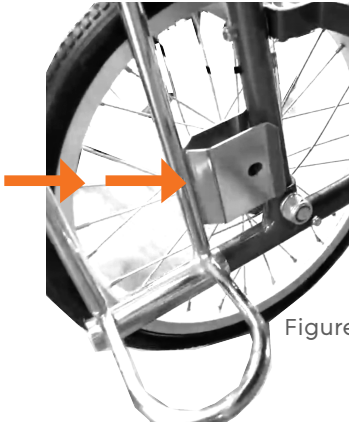


Figure 1

**Tools needed:**  
5mm Hex Key

### Step 1

Remove the screw attached to the attachment plate. Position the cart flush to the left wheel of the wheelchair as pictured (Figure 1).

### Step 2

Using the hex key, attach the screw to the attachment plate until it is securely attached (Figure 2).

### Step 3

Place oxygen tank into cart. Tighten black knob to secure tank in the cart. Check that the cart and oxygen tank are secure before using wheelchair.



Figure 2



Figure 3