
Getting Started Guide Guía de Inicio

Portable Oxygen Concentrator Concentrador de Oxígeno Portátil

LANGUAGE SELECTION (ENGLISH OR SPANISH) DEFAULT LANGUAGE IS ENGLISH.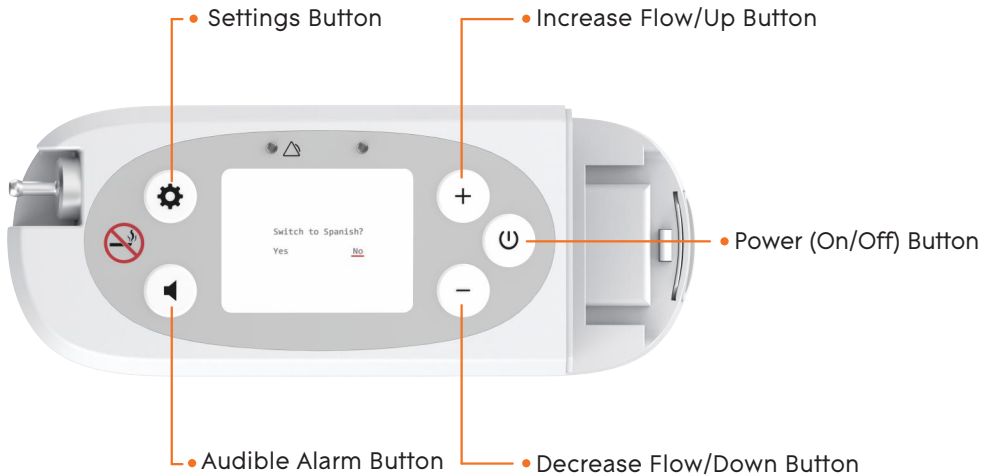

1. Press Power button to turn on POC.
2. Press the Settings button. The default language is English.
3. Press and hold the Settings button again for 5 seconds.

The interface will display: Switch to Spanish?

4. Then press the - or + to select Yes/No.
5. Press the Settings button to confirm Yes/No.
 - Confirm "Yes" to switch to Spanish.
 - Confirm "No" to confirm English.
6. Press the Settings button.

Return to the device startup interface, then you can use POC normally.

| CHECK FILTER REMINDER


When accumulated runtime has reached 600 hours, the Check Filter screen will appear.
Scan the QR code for instructions.

Press left arrow to check filter later.

Press right arrow if filter has already been checked and cleaned.

Device will then enter startup process and reset runtime.

Note: If you do nothing, the Check Filter screen will appear the next time you power on device.

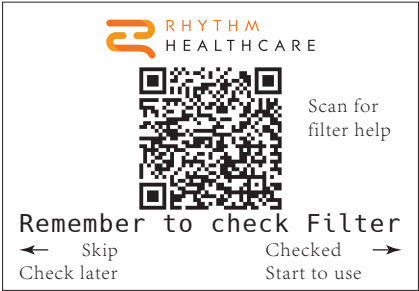| STARTUP ERROR ALARM

An error will be displayed after startup if alarm persists more than three times consecutively during each startup.

Press left arrow to check later.

Press the right arrow if error has already been checked.

Device will then enter startup process.

QUICK SETUP INSTRUCTIONS

1

Find a well ventilated location to place the P2. Ensure the intake and exhaust have clear access.

2

Slide the flat side of the battery until the latch returns to the upper position and you hear it click into place.

3

Plug the power supply AC cord into the nearest electrical outlet.


4

Connect the AC power to the P2. The green LED on the power adapter will be on and the concentrator will beep.

QUICK SETUP INSTRUCTIONS - CONTINUED

5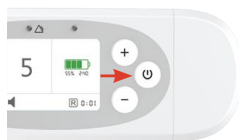


Press the On/Off button, device will beep and the indicator light will flash. A two minute warm-up time will initiate.

6

Connect the nasal cannula to the nozzle fitting at the top of the P2. Ensure that it is routed to prevent it from being pinched or kinked to avoid disruption of oxygen flow.

7

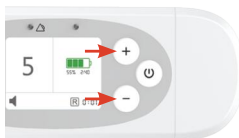Change the P2 to the flow setting prescribed by your provider. Press the + or - buttons to adjust to the desired setting.

8

Press and hold the On/Off button for 3 seconds to shut down the P2.

Additional information:

When the adapter is connected or disconnected, or the battery is installed or removed, the device will beep and display the battery charge status.

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Rhythm Healthcare, LLC