RHYTHM  
HEALTHCARE

TOC3

# QUICK START GUIDE GUÍA DE INICIO

---

Transportable Oxygen Concentrator  
Concentrador de Oxígeno Transportable


# TOC3

Now that you have unpacked your box, you have everything you need to start using your TOC3. The setup of your TOC3 is quick and simple, however, before using your new TOC3, please read the entire user manual.

## LANGUAGE SELECTION (ENGLISH OR SPANISH) DEFAULT LANGUAGE IS ENGLISH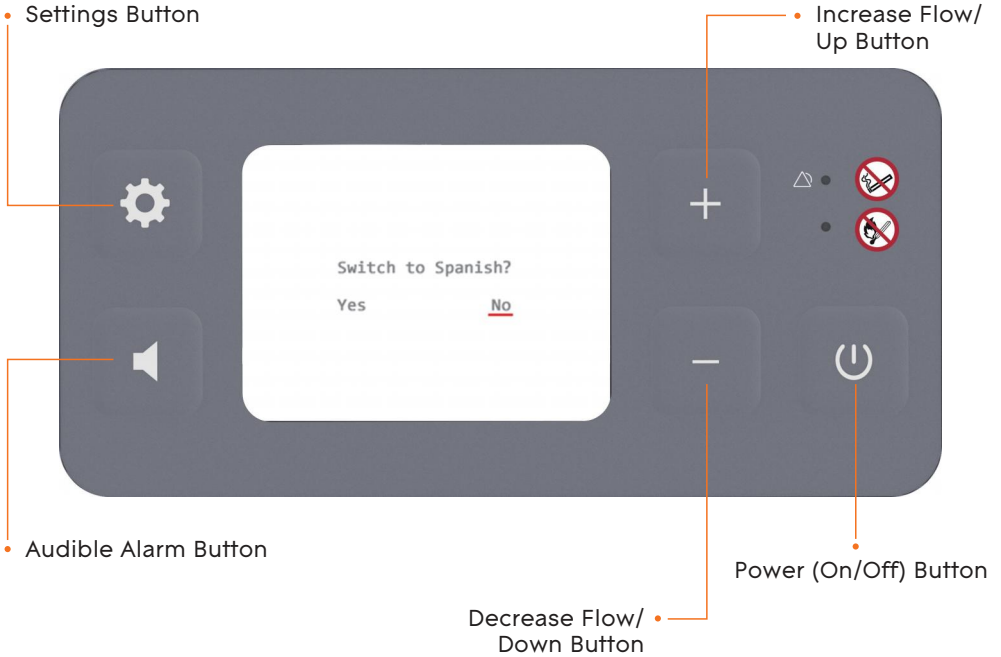

1. Press Power button to turn on TOC3.
2. Press and hold the Settings button. The default language is English.
3. Press and hold the Settings button again for 5 seconds.  
The interface will display: Switch to Spanish?
4. Then press the - or + to select Yes/No.
5. Press the Settings button to confirm Yes/No.
  - Confirm "Yes" to switch to Spanish.
  - Confirm "No" to confirm English.
6. Press the Settings button.  
Return to the device startup interface, then you can use TOC3 normally.

## | CHECK FILTER REMINDER

When accumulated runtime has reached 600 hours, the Check Filter screen will appear.

Scan the QR code for instructions.

Press left arrow to check filter later.

Press right arrow if filter has already been checked and cleaned.

Device will then enter startup process and reset runtime.

Note: If you do nothing, the Check Filter screen will appear the next time you power on device.


## | STARTUP ERROR ALARM

An error will be displayed after startup if alarm persists more than three times consecutively during each startup.

Press left arrow to check later.

Press the right arrow if error has already been checked.

Device will then enter startup process.

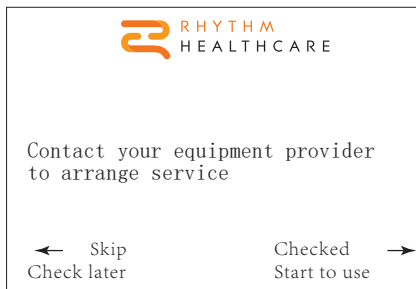## QUICK SETUP INSTRUCTIONS

1

Find a well ventilated location to place the TOC3. Ensure the intake and exhaust have clear access.

2

Slide the flat side of the battery until the latch returns to the upper position and you hear it click into place.

3

Plug the AC Power Cord into the nearest electrical outlet.

4

Connect the other end of the AC Power Cord to the TOC3 power port.


5

Connect the Nasal Cannula to the Nozzle Fitting at the top of the TOC3. Ensure it is routed to prevent it from being pinched or kinked to avoid disruption of oxygen flow.


## QUICK SETUP INSTRUCTIONS - CONTINUED

6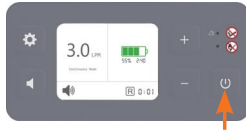


Press the On/Off button.  
The device will beep and the indicator light will flash. A five minute warm-up time will initiate.

7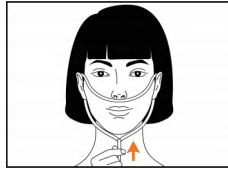

Wear the Nasal Cannula and breathe through your nose.

8

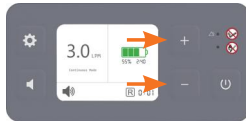**To Select the Flow Setting Prescribed by your Provider:**  
Press the Settings button to switch between Pulse and Continuous Modes. Press the + or - buttons to adjust the flow setting.

9

**To Shut Down the TOC3:**  
Press and hold the On/Off button for three seconds.

### Additional information:

When the adapter is connected or disconnected, or the battery is installed or removed, the device will beep and display the battery charge status.

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Rhythm Healthcare, LLC