



rhythmhc.com | contactus@rhythmhc.com | (877) 843-6464

Please read and understand these instructions
before using the charger.

Includes: External Battery Charger and AC Adapter

The external battery charger is designed to power the battery
for P2-S3 and P2-S4. Use the External Charger and
Portable Oxygen Concentrator as directed.

The AC adapter included will also power the P2-S3 and P2-S4 devices.




Charging Time

Single Battery: Up to 2 hours

Double Battery: Up to 4 hours

Triple Battery: Up to 5 hours

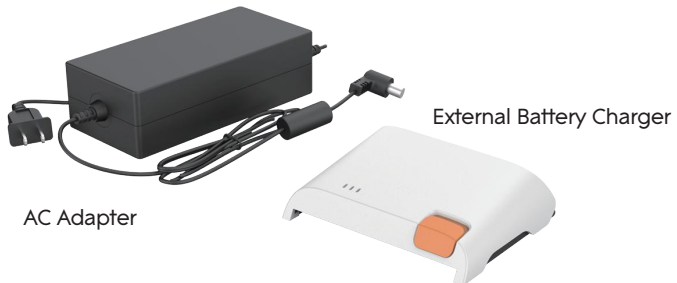

1. Plug the AC adapter into an outlet and the other end to the external charger. The indicator lights will flash red, green and blue. Then the solid red light will light up.

2. Slide the battery into the charger until you hear it click into place.



3. Flashing blue light indicates that the battery is charging.

4. Green solid light indicates that the battery is fully charged.



5. Disconnect the power supply and the AC adapter before storing the charger.