



ITEM# B3800F

BARIATRIC FOLDING COMMODE

650 lb
Weight Capacity



Rhythm Healthcare,
Clearwater, FL 33762

contactus@rhythmhc.com
www.rhythmhc.com
877 843 6464

V2: 10/2/23

PARTS DIAGRAM




ASSEMBLY INSTRUCTIONS

! If Commode Seat has been exposed to cold, allow it to reach room temperature before assembly to prevent cracking. Please read and understand manual prior to assembly and save for future use.

To Set Up Commode Frame

1. Unfold frame. Make sure sides of frame are at right angles to the rear of the frame and the open ends (of cross brace sockets) are facing center of frame.
2. Swing the bucket support frame up and insert the ends of cross brace into the cross brace sockets on each side frame by depressing the push pins until they click securely into place.
3. Insert the back brace into back of frame. Ensure push pins click securely into place.
4. Attach commode seat and lid to rear of frame by pressing clips onto rear cross brace until they snap securely in place.

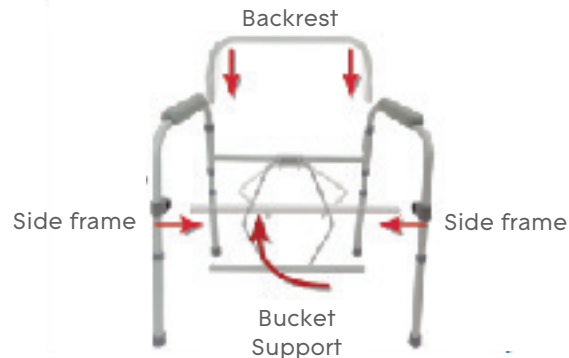

OPERATING INSTRUCTIONS

To Adjust Legs

- 1 Depress the push pin and slide the lower leg up or down to desired height until push pin clicks into place.
- 2 Repeat step 1 for each leg, ensuring they are set at same height.

Using as Bedside Commode

- 1 Lift commode seat and place bucket on support rails, making sure it is directly below the opening. Lower seat for use.
- 2 When not in use (to contain the odor), place the lid on the bucket and use handle on bucket for cleaning.

Using Commode as Toilet

- 1 Adjust height, if necessary, and remove the back brace. Then replace bucket with splash guard.
- 2 Lift toilet bowl seat and place commode over toilet, center the splash guard directly over the toilet bowl. Lower commode seat for use.

SAFETY PRECAUTIONS

! Rhythm Healthcare, LLC is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.

- > If you do not understand these instructions, contact your dealer or a healthcare professional for assistance.
- > If any parts are damaged or missing, contact your dealer for replacement. Do not substitute parts.
- > Ensure commode seat is securely attached to the frame and all legs are set at the same height before use.
- > Users with limited physical strength will require assistance/supervision.
- > Weight capacity is 300 lb, evenly distributed.

MAINTENANCE

- > Clean after each use with disinfectant.
- > Do NOT use organic solvents, such as acetone or abrasive cleaners.
- > Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or replacement parts.

WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended, but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm Healthcare, LLC, prior authorization is to be required. In the event of a defect covered by this Warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced. Rhythm Healthcare, LLC products are identified by a serial or a lot number label at the time of the manufacture. Removing or defacing the label may void this warranty. If you have questions about your Rhythm Healthcare, LLC product or this warranty, please email us at contactus@rhythmhc.com