


P2-E6 PORTABLE OXYGEN CONCENTRATOR PARTS

DESCRIPTION	DIAGRAM #	RHYTHM ITEM #
Pre-Filter Cover	1	P2E6PFCR
Nozzle	2	P2NZL-2
Sealing Ring	3	P2SLR-2
st2.2*5 304 Stainless Steel	4	P2SC225-2
Cable Cover on Main Case	5	P2CCMC-2
Display Panel (Keyboard)	6	P2E6DP
Cotton Intake Filter	7	P2FC-1
Pack for Cooler and Fixed Mount Accessories	8	P2E6PCFM
L-type Silicone Hose	9	P2LSH-2
Plastic Nail	10	P2PNL-1
Air Outlet Seat	11	P2AOS-2
Leather Cup	12	P2LRC-2
Main PCB with Screen	13	P2E6PCB
st2.9*6 304 Stainless Steel	14	P2SC296-2
PU Tube	15	P2PUTB425-2
Motherboard Power Cord	16	P2MPC-2
Heat Shrinkable Tube	17	P2HSTB945-2
Motherboard Signal Line	18	P2MSL-2
Side Power PCB	19	P2SPB-1
Front Solenoid Valve Control Line	20	P2E6FSV
Structural Plate	21	P2STP-2
Battery	22	P2BY001-2
Sieve Bed Compartment Cover	23	P2E6SBCC
Solenoid and Base Pack	24	P2E6SDBP

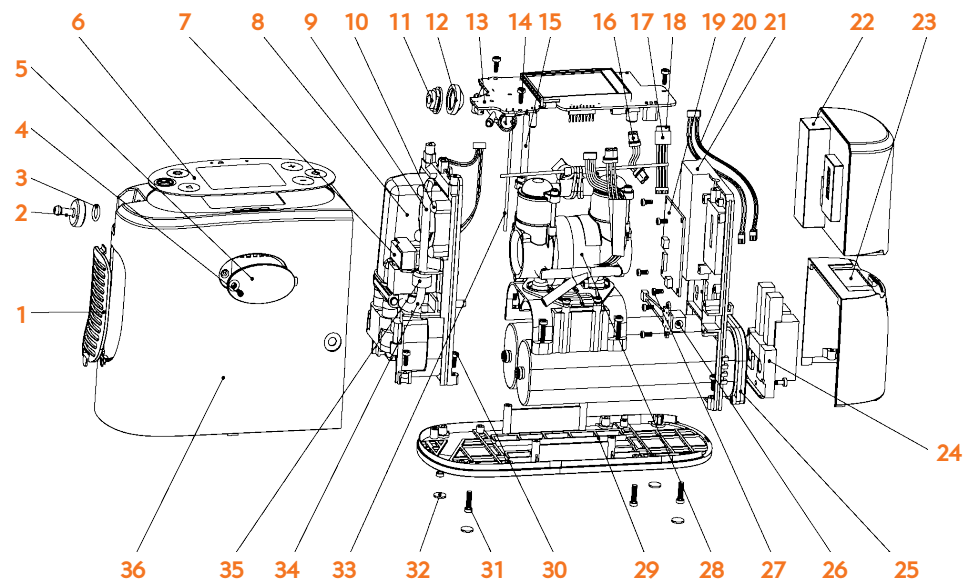

DESCRIPTION	DIAGRAM #	RHYTHM ITEM #
Sieve Bed	25	P2E6SB
Adapter Terminal Block	26	P2ATB-2
st2.6*6 304 Stainless Steel	27	P2SC266-2
Compressor	28	P2E6C
Bottom Base	29	P2BMB-2
st2.9*8 304 Stainless Steel	30	P2SC298-2
st2.9*12 304 Stainless Steel	31	P2SC2912-2
Anit Slip Mat	32	P2ASM-1
Silicone Tube: Diameter 2mm, Inner Diameter 0.95mm	33	P2STB2095
Intake Elbow	34	P2IKE-2
Filter Assembly	35	P2FTA-2
Main Case	36	P2MNC-2

P2-E6 PORTABLE OXYGEN CONCENTRATOR DIAGRAM

