
Getting Started Guide Guía de Inicio

Portable Oxygen Concentrator Concentrador de Oxígeno Portátil

LANGUAGE SELECTION (ENGLISH OR SPANISH) DEFAULT LANGUAGE IS ENGLISH.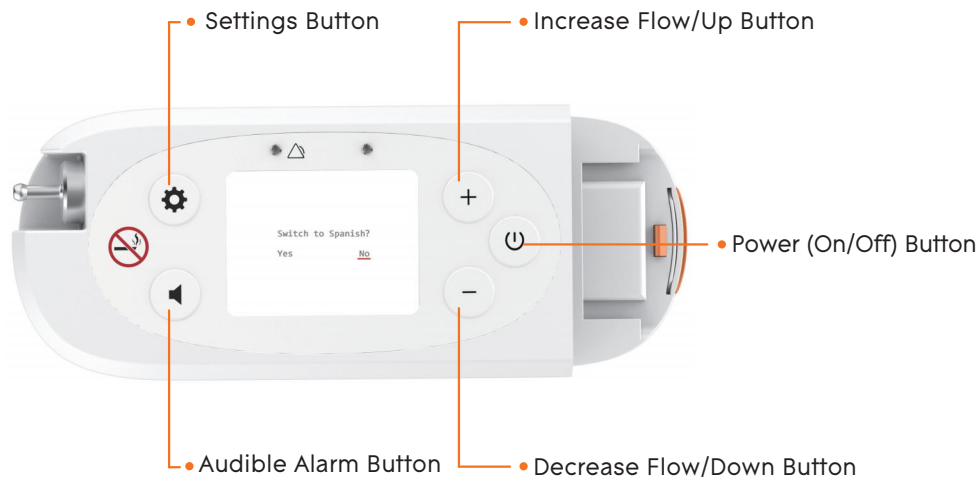

1. Press Power button to turn on POC.
2. Press the Settings button. The default language is English.
3. Press and hold the Settings button again for 5 seconds.

The interface will display: Switch to Spanish?

4. Then press the - or + to select Yes/No.
5. Press the Settings button to confirm Yes/No.
 - Confirm "Yes" to switch to Spanish.
 - Confirm "No" to confirm English.
6. Press the Settings button.

Return to the device startup interface, then you can use POC normally.

| CHECK FILTER REMINDER

When accumulated runtime has reached 600 hours, the Check Filter screen will appear.


Scan the QR code for instructions.

Press left arrow to check filter later.

Press right arrow if filter has already been checked and cleaned.

Device will then enter startup process and reset runtime.

Note: If you do nothing, the Check Filter screen will appear the next time you power on device.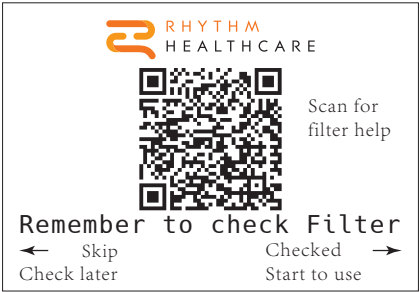

| STARTUP ERROR ALARM








An error will be displayed after startup if alarm persists more than three times consecutively during each startup.

Press left arrow to check later.

Press the right arrow if error has already been checked.

Device will then enter startup process.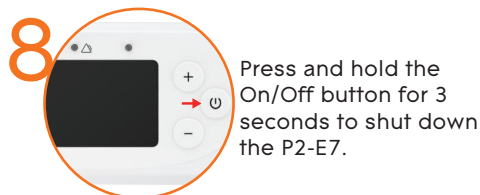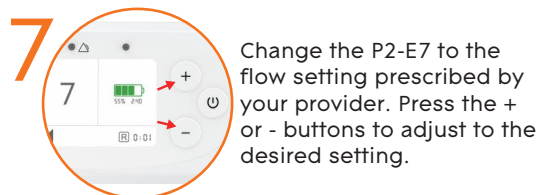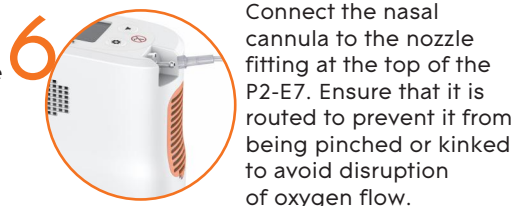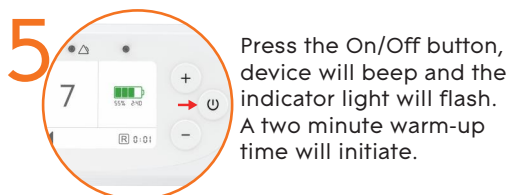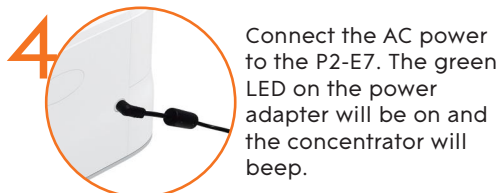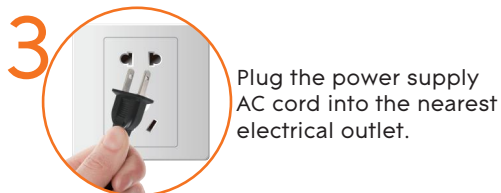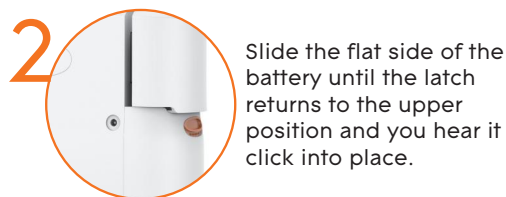

Additional information:

When the adapter is connected or disconnected, or the battery is installed or removed, the device will beep and display the battery charge status.

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Rhythm Healthcare, LLC